**Table des Matieres**

**Content**

**Inhalt**

**Contenuto**

**Indice**

**Inhoud**

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
17

## Content

<b>Items supplied</b>	<b>20</b>	<b>7 Measurement of pulse rate and blood pressure</b>	<b>26</b>
<b>Items available on request</b>	<b>20</b>	<b>8 Memory function</b>	<b>26</b>
<b>1 Information on blood pressure</b>	<b>20</b>	<b>8.1 Saving the reading</b>	<b>26</b>
<b>2 Technical data</b>	<b>21</b>	<b>8.2 Memory recall</b>	<b>27</b>
<b>3 Please read carefully before use!</b>	<b>22</b>	<b>8.3 Memory deletion</b>	<b>27</b>
3.1 Recommended use	22	<b>9 Care and maintenance</b>	<b>27</b>
3.2 Warnings	22	9.1 Care and maintenance	27
<b>4 Functional components</b>	<b>23</b>	9.2 Changing the arm cuff	28
4.1 Device description	23	<b>10 Descriptions of errors</b>	<b>28</b>
4.2 LCD Display description	23	10.1 Error numbers	28
<b>5 Installation</b>	<b>24</b>	10.2 Errors classifications and corrections	29
5.1 Preparation	24	<b>11 Warranty and repair conditions</b>	<b>30</b>
5.2 Set date and time	24	11.1 Warranty	30
<b>6 Use of the "Pressio A"</b>	<b>25</b>	11.2 Repair	30
		<b>12 Legislation</b>	<b>30</b>

18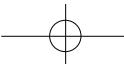

19

This adjustable arm cuff of your "Pressio A" for pulse and blood pressure measurement provides a top-quality system incorporating the latest technology, which is easy to use and extremely reliable. Thank you for your confidence; please do not hesitate to contact us at any time if you have queries concerning the product or servicing.

### Items supplied

1. "Pressio A" table top pulse and blood pressure measurement device with arm cuff for home use
2. Instructions for use
3. Four batteries (4 x 1.5V AA size alkaline)
4. Arm cuff regular size approx. 23 – 33 cm

### Items available on request

1. Arm cuff regular size approx. 33 – 43 cm
2. AC Adapter Jack

### I Information on blood pressure

The circulatory system in our bodies supplies all organs and tissues with sufficient oxygen and nutrients to meet their requirements. The heart acts as a pump to keep the blood moving. It normally beats to a regular rhythm, 60 to 80 times a minute. This is known as the heart rate or pulse rate.

Blood pressure is subject to a number of changes in the course of the day and during the night. When sleeping, blood pressure is low. High to very high values occur when we are agitated or in periods of physical exertion. The temporary increase in blood pressure at these times is quite normal. It soon goes down again.


Blood pressure is the result of contraction and relaxation of the beating heart. The upper (systolic) and lower (diastolic) pressures can be measured.

Blood pressure readings are expressed in millimetres of mercury (mmHg).

**The upper limit of the normal range for adults is 135 / 85 mmHg.**  
(recommendations for the measurement of blood pressure from the Deutsche Liga zur Bekämpfung des hohen Blutdrucks e.V., Heidelberg)

If we measure blood pressure regularly, we can learn a great deal about the way our own bodies work and their reactions to outside influences.  
We can therefore lead healthier lives!

Why not find out your own blood pressure patterns? Your doctor will certainly be pleased to help. Please find below a first overview which is subject to further interpretation by a qualified medical person.

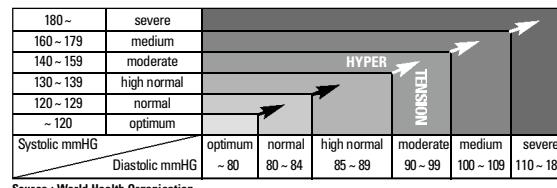### 2 Technical data

#### Range of measurement

Blood Pressure: 30–280 mmHg

Pulse Rate: 40–200 beats/min

#### Measurement method

Oscillometric method

**Storage capacity** 60 measurements  
(systolic and diastolic pressure, pulse rate, date and time)

#### Accuracy Calibration

Blood Pressure:  $\pm 3$  mmHg

Pulse rate: +/- 4%

**Power source** 4 x 1.5V AA size alkaline batteries (DC6V)  
(AC Adapter Jack available on request)

**Display** Liquid crystal display (LCD)

Visible size approx. 70 x 50 mm

**Operating temperature** +10 °C – +40 °C

**Operating humidity** 15% – 95% RH (non-condensing)

**Storage temperature** -20 °C – +50 °C

**Storage humidity** 15% – 95% RH (non-condensing)

**Cuff size** Arm Cuff circumference between approx. 23 and 33 cm  
(Arm Cuff circumference between approx. 33 and 43 cm available on request)

**Dimensions** 128 mm x 150 mm x 59 mm (W x H x D)

**Weight** Approximately 390 g (excluding batteries)

### 3 Please read carefully before use!

#### 3.1 Recommended use

- The "Pressio A" is to be used in measuring pulse rate and blood pressure. To obtain accurate values, the blood pressure monitor should be worn on the arm under any clothing. **During the measurement, do not move and keep silent to ensure best accuracy. It is also important that your pressure cuff is at the same height level as your heart.**
- Avoid measuring blood pressure in 30 minutes after exercise, bathing, or returning from outdoor.
- To switch off your "Pressio A" or stop any current action, please press "START/STOP" button. If users forget to switch off the device, it will turn off automatically after two minutes idling.


#### 3.2 Warnings

- Never make any change to your current medication on the basis of the results you obtain. The assessment of results and decisions regarding subsequent treatment can only be taken by the doctor.**
- If you suffer from cardiac arrhythmias (irregular Heartbeats) or are pregnant you should consult your doctor to check whether this method of measuring your own blood pressure with the "Pressio A" is suitable for you.
- This unit is not suitable for use in cases of severe arteriosclerosis, advanced arterial occlusive disease or in conditions involving vascular spasm.
- If discomfort is experienced during a measurement, press the START/STOP key to deflate the arm cuff immediately. Unfasten the arm cuff and remove it. Please contact your dealer.
- The unit should not be operated in the vicinity of an MRI scanner.
- The device is suitable for measuring pulse and blood pressure in adults only.
- The pulse display on the unit cannot be used for checking the heart rate of a cardiac pacemaker.
- The use of components not supplied with the equipment can lead to incorrect measurements (except for batteries).
- On no account use the "Pressio A" when bathing or showering. Should moisture penetrate the unit, the batteries must be removed immediately and further use of the unit avoided. Contact your dealer.

## 4 Functional components


"Pressio A" Upper-Arm Blood Pressure Monitor applies the Oscillometric method to measure human systolic/diastolic blood pressure and pulse rate. All values are shown on the LCD monitor.

#### 4.1 Device description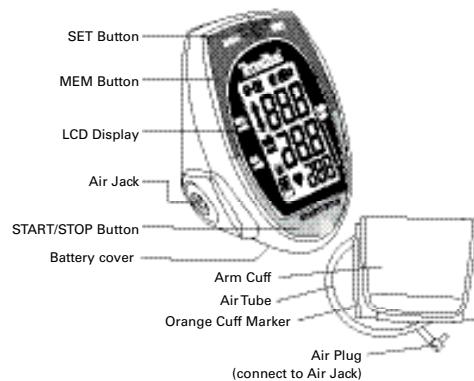

#### 4.2 LCD display description


- |                              |                              |
|------------------------------|------------------------------|
| ① Date                       | ② Time                       |
| ③ Systolic Pressure          | ④ Diastolic Pressure         |
| ⑤ Pulse Symbol               | ⑥ Fréquence cardiaque        |
| ⑦ Low Battery Symbol         | ⑧ Memory Set                 |
| ⑨ Memory Symbol              | ⑩ Average Measurement Symbol |
| ⑪ Inflation/Deflation Symbol |                              |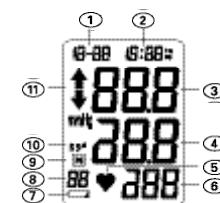

## 5 Installation

### 5.1 Preparation


Open the battery compartment on the back of the device. Insert the batteries supplied as shown in the diagram. Then close the cover.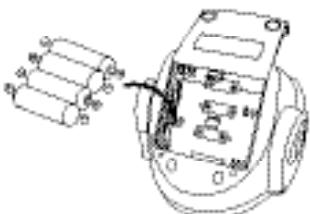

### 5.2 Set date and time

- While in power off mode, press the "SET" button to enter Date/Time setting procedure and the Month value will begin to flash.
- To adjust the value of Month, press "MEM" button. Pressing once will advance the flashing value by 1. If the "MEM" button is held down, the value increases automatically.
- Once finished, press "SET" button again to confirm the set value and go to the next setting.

Follow the steps instructed above to adjust the value of date and time consecutively.

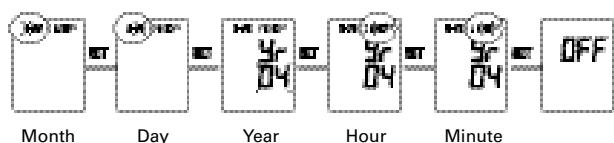
## 6 Use of "Pressio A"


- Make sure the air plug is securely inserted in the main unit as shown in the diagram.

- Remove tight-fitting clothing from your upper arm.
- Turn the palm of your hand upward. Apply the arm cuff on your upper arm. Secure the arm cuff around your arm using the velcro strip.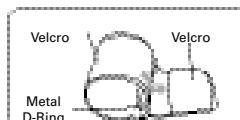

- Make sure the arm cuff is wrapped firmly in place.
- The bottom of the arm cuff should be approximately 2 to 3 cm above your elbow. Make sure the orange marker is in your inner arm and "Index" is in the range of

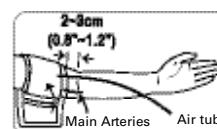" ← OK → "


- Sit in a chair with your feet flat on the floor and place your arm on table so that the arm cuff will be at the same level as your heart.

Blood pressure naturally differs between two arms, so always measure your blood pressure on the same arm, in order to ensure comparability of two readings.

## 7 Measurement of pulse rate and blood pressure


- Press the "START/STOP" button.
- All display symbols appear for approx. 2 seconds. This is the initial LCD test.

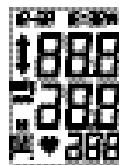- When the monitor completes the necessary preparations before measurement, the last reading result will be shown on the display for 1 second.

- Then the monitor begins to measure your blood pressure, and you will see the inflation symbol on the display when inflating. During the process of deflation, the symbol of inflation will turn off and the symbol of deflation will turn on instead.

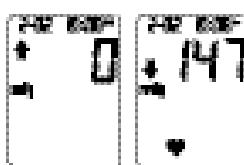- After the measurement is completed, your blood pressure and pulse will be displayed on the LCD automatically.

## 8 Memory function

### 8.1 Saving the reading

- Your "Pressio A" can store data from up to 60 measurements.
- Once the 60 measurements are recorded, each subsequent reading replaces the "oldest" in the memory.

### 8.2 Memory recall

- To recall the memory, please press "MEM" button first. Then, the average result of all stored measurements will be shown on the LCD first.
- After the average reading is shown on the display, you can press "MEM" button to track historical records from no.1 to no.60.

- To come back to the first records, please press "SET" button.

**This function is available only after readings have been saved to memory!**  
If no data has been stored, the display will go directly to the "setting date and time" function.

### 8.3 Memory deletion

While in power off mode, press "MEM" button and hold it until the screen shows "Clr".

## 9 Care and maintenance

### 9.1 Care and maintenance

- If necessary, clean the device only with a slightly moistened cotton cloth or mild detergent.
- Do not disassemble, repair or remodel the monitor.
- Do not expose the monitor to extreme temperature, very high humidity, or direct sunlight.
- Avoid dropping and rough handling.
- Do not immerse the device in water. It may cause damage if water enters.
- Do not use any gas, strong detergent, or solvents to clean the device, including the arm cuff.

- **Batteries should not be discarded with domestic waste!**  
Please dispose of batteries only in the disposal bins provided in shops.

### 9.2 Changing the arm cuff

If you need a new arm cuff for your "Pressio A", this can be obtained from Terraillon as an accessory. **For technical reasons, the arm cuff should only be changed by Terraillon.** Please consult your dealer if necessary.

## 10 Descriptions of errors

### 10.1 Error numbers

Numbers	Causes
Err 0	No pulse or detected pulse is out of range.
Err 1	Arm cuff serious leakage or inflation too low.
Err 2	Systolic cannot be measured.
Err 3	Arm cuff pressure is too high.
Err 4	Diastolic cannot be measured or is outside of range.
Err 5	Deflation is too fast.
Err 6	Deflation is too slow.
Err 7	Systolic outsides the legal range.
Err 8	Mean pressure is outside of range.
Err 9	Arm cuff pressure is not enough even if the highest value is pumped.

### 10.2 Errors classifications and corrections

Conditions	Causes	Corrective Actions
No display after push the "START/STOP" button	The position of the batteries is incorrect	Insert the batteries in the correct position (e.g 5.1)
	The voltage of the batteries is extremely low	Replace both batteries with new ones.
Low battery symbol appears	The voltage of the batteries is extremely low	Replace both batteries with new ones.
	When chilly, the batteries might have a weak electron release	Warm up the batteries, or place in a warm place for a while then re-start again
Battery life is too short.	May be using wrong battery type.	Use only Alkaline batteries.
No reading after measurement	The voltage of the batteries is extremely low	Replace both batteries with new ones
Suspicious measurement results. Higher or lower than user's average measurement	May not be in correct position during measuring	Always take measurements in correct position (e.g 6)
	The blood pressure naturally varies from time to time	Relax and try to measure again
Pulse rate higher/lower than user's average.	Move arm or body during measurement	Do not move arm or body during measurement
	Measuring right after exercise	Do not measure right after exercise
Power switches off automatically	System design	Push the "START/STOP" button again, and then measure again
Device reinflates during measurement.	This is normal: if user's blood pressure exceeds the initial pressure value, the device will automatically pump to a higher pressure, by 20mmHg each time, until it reaches a suitable level.	Relax and try to measure again
	The arm cuff is not fastened	Fasten the arm cuff again

## II Warranty and repair conditions

### 11.1 Warranty

- Terraillon provide a two year warranty on the blood pressure measuring unit from the date of purchase. The correctly completed warranty card or invoice constitute evidence of the date of purchase.
- Deficiencies due to material defects or manufacturing errors will be corrected free of charge within the warranty period.
- Services provided under warranty do not extend the period of warranty, either for the unit itself or for selected components.
- The following are excluded from the warranty:
  - all damage resulting from incorrect handling, e.g. failure to observe the instructions for use.
  - damage attributable to repair or intervention by the retailer or unauthorised third parties.
  - damage occurring in transit between the manufacturer and the user or during return to the customer service department.
  - accessories subject to normal wear and tear (arm cuff, batteries, etc.).
- Liability for direct or indirect damage caused by the equipment is also excluded even if the damage to the unit is recognised as covered by the warranty.
- Lost warranty cards cannot be replaced. Warranty cards altered without justification are invalid.
- Further claims, for whatever reasons these are made, are excluded.

Terraillon provide no warranty for the batteries supplied with the equipment.

### 11.2 Repair

In the case of a warranty claim, please complete the warranty card with your exact name and address and full details of the fault. Please contact your dealer for information on methods of dispatch.

## 12 Legislation

"Pressio A" conforms to the European specifications on which medical products legislation is based. The quality of the equipment has been examined under EC regulation 93/42/EEC and bears the mark CE0197.

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### Warranty Card

For service and repair contact

Address/Stamp of distribution/Dealer

If not stamped, contact your point of purchase for further information.

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