Movement Sensor Pad Monitor TSP500

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Movement Sensor Pad Monitor TSP500

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
• GETTING TO KNOW YOUR MONITOR

GB

Your Tomy Movement Sensor Pad Monitor TSP500 has been carefully designed to sit underneath the mattress of your cot or Moses basket to detect all movements baby makes and to emit an alarm when no movement is detected after 20 seconds.

Every movement is accompanied by a soft click and an illuminated visual indicator. Should no movement be detected on the Movement Sensor Pad for 20 seconds, a continual beep alarm will be activated.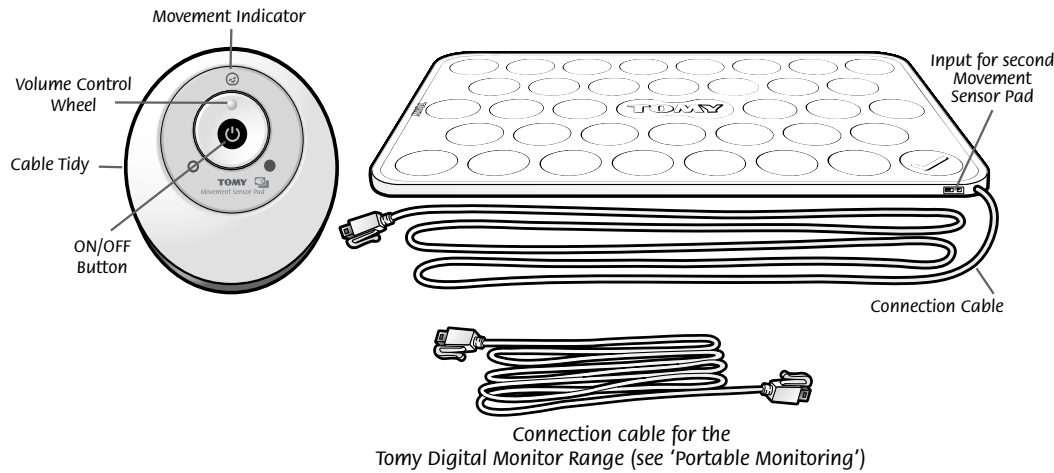


• SET UP

• POSITIONING THE MOVEMENT SENSOR PAD TSP500

The Tomy Movement Sensor Pad TSP500 sits underneath the mattress of your cot or Moses basket as shown (below). Once you have chosen baby's sleeping area, position the Movement Sensor Pad to lie directly underneath her torso.

IMPORTANT! The Movement Sensor Pad must rest on a completely flat, rigid surface and must be placed with the 'Tomy logo' side facing up. Ensure that no bedding is positioned between the mattress and the Movement Sensor Pad.


For best performance we recommend covering the base of slatted based cots with a thin layer (6mm) of plywood to ensure an even surface. Ensure any loose cable is wrapped and tied to avoid possible entanglement. From the cot, run the cable along the floor ensuring it is kept out of baby's reach; excess cable should be spooled around the cable tidy.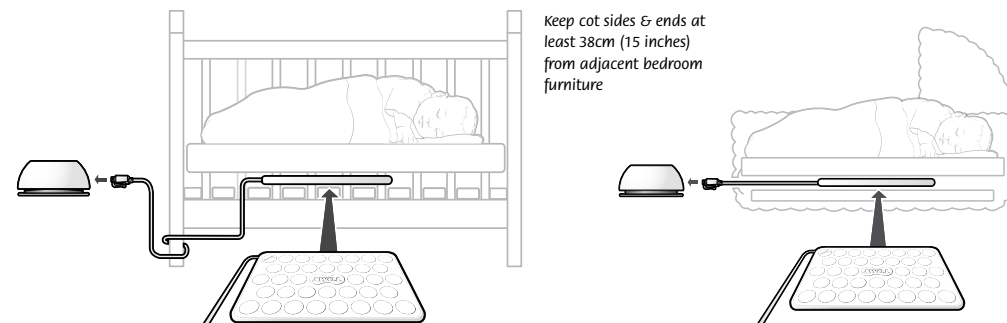

• FEATURES AND FUNCTIONS


GB

• PAD CONTROLLER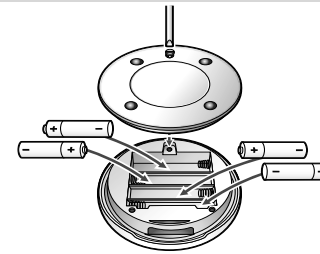

The Pad Controller is solely battery operated and requires 4 x AA (LR06) batteries (not supplied). Insert batteries by removing the battery compartment cover on the underside of the unit using a flat headed screw driver. Insert the batteries ensuring correct polarity and replace the cover, screwing firmly into position. To activate the Pad Controller, press and hold the button until it illuminates green.

• CONNECTING THE PAD TO THE CONTROLLER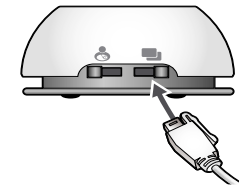

Connect the Movement Sensor Pad cable to the socket on the Pad Controller. To ensure that the product is as reliable as possible Tomy have developed locking connectors for the cables between the Pad and Pad Controller (and Pad Controller and Portable Monitor). When connecting, push the connector plug into the socket until the locking arm of the plug clicks. To release the plug, firmly squeeze and pull the plug so that the locking arm is released from the socket.

• TESTING

Before every use we recommend carrying out the following checks with the Pad in the cot or Moses basket (as per the Set Up guide). Firstly tap the mattress, at which time the Pad Controller should emit a soft click and the indicator shall flash Green. Follow this by leaving the Pad to stand untouched for 20 seconds, then the alarm shall sound and the indicator shall illuminate red. Simply deactivate the alarm by tapping the mattress again.

• PAD CONTROLLER VOLUME WHEEL

The volume of the soft click can be adjusted using the volume control wheel on the Pad Controller from mute to max. Move the thumb wheel clockwise to increase or anti-clockwise to decrease.

You can also monitor baby's movement without the movement click by turning the Volume Control Wheel to the mute position, and observing the movement indicator on the Pad Controller that also accompanies every movement (just like the click).

Please be assured that during mute mode, the no movement alarm shall be unaffected and the continual beep alarm will sound if no movement is detected for 20 seconds.

• LOW BATTERY ALARM


The Pad Controller will emit a 4 'beep' alarm every 60 seconds and the button shall flash red, indicating that the batteries need replacing.

• ADDITIONAL MOVEMENT SENSOR PAD

To monitor a larger surface area within the cot, the supplied Movement Sensor Pad has been designed to connect to another Tomy Movement Sensor Pad. For further details, please visit our website; www.tomy.eu

• PORTABLE MONITORING

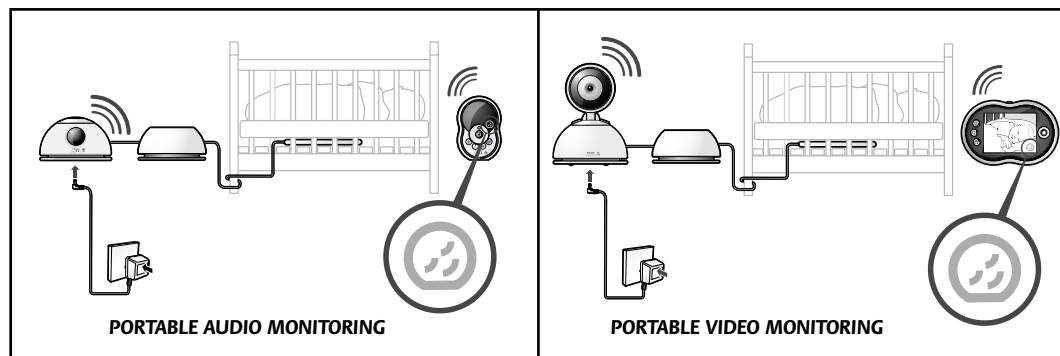The Tomy Movement Sensor Pad TSP500 can be used portably with any Tomy Audio or Video monitor and has been specifically designed to connect to the Tomy TD and TDV Monitor Series. This allows for the portable monitoring of movement, sounds and video from afar, with the added features of: a moving sound level lights display; mains (AC) operated power for all units (including the Movement Sensor Pad); a soft glow/multicoloured nightlight; monitoring baby's room temperature - as well as many other features specific to each individual model.

• Connecting to a Tomy TD or TDV Series Monitor

With the Movement Sensor Pad in position within the cot or Moses basket and your chosen portable monitor operating as per its supplied instruction booklet, take the connection cable (supplied with the Tomy Movement Sensor Pad monitor) and plug one end into the  socket on the Pad Controller and the other end into the  socket on the Baby Unit of the portable monitor.

All sounds and alarms shall automatically transfer to the Parent Unit of your portable monitor. This will be signalled by a beep emitted by the Parent Unit of the portable monitor (so that your baby is not disturbed).

Powering the Pad Monitor

The Pad Controller will draw power from the connected Baby Unit only if the Baby Unit is powered by the Main (AC). If the Baby Unit is powered by Batteries (DC) the Pad Controller will continue to draw power from its own battery supply (or will not function if none are present or they are flat).

If powering the Baby Unit via mains and therefore the Pad Controller by mains, **we recommend installing a new set of batteries within the Pad Controller so that in the event of a power failure, monitoring continues.**

Fail Safe

In the event of a connection failure (possibly caused by a power failure or loss of signal etc. - please refer to Fault Finding guide), all Movement Sensor Pad monitoring shall be transferred back to the Pad Controller, as long as it has battery power.

Using a Tomy non TD or TDV series Monitor

With the Movement Sensor Pad in position within the cot or Moses basket place the Baby Unit of the portable monitor within 20cm of the Pad Controller. Ensuring the volume of the Pad Controller is set to an audible level use the Parent Unit to listen for the soft click and alarms.

• WARNINGS

THE TOMY MOVEMENT SENSOR PAD IS NOT A MEDICAL DEVICE DESIGNED TO PREVENT CASES OF SUDDEN INFANT DEATH SYNDROME (SIDS).

The Tomy Movement Sensor Pad is NOT a substitute for direct supervision of your baby. Check your baby's activities at regular intervals. Monitoring of premature babies, or those considered to be at risk, should be carried out under the supervision of a doctor or health professional.

The Tomy Movement Sensor Pad may detect movement from many sources from both inside and outside your baby's room, for example from fans, a washing machine, loud music, etc. Make sure that all vibration sources are eliminated before using the Tomy Movement Sensor Pad monitor.

Always place both the Movement Sensor Pad and Pad Controller on a flat surface in the upright position, away from baby's reach.

When the monitor is in use, DO NOT use a cot mobile or other cot accessory that may produce vibrations; the alarm will not sound as the monitor will still be detecting movement.

The Tomy Movement Sensor Pad is perfectly safe. The Movement Sensor Pad and its cord are entirely passive; they carry no electrical current and do not radiate any form of energy.

The Tomy Movement Sensor Pad can be used in any cot or sleeping area (that has no motion or vibration), designed to safely hold a baby.

250 metres achievable in optimum open field conditions.

• CARING FOR YOUR MONITOR

Do not immerse any parts in liquids. Do not place in direct sunlight.

Periodically examine product for signs of damage to electrical parts and do not use until damage has been properly repaired. Always remove batteries from the Baby Unit if using the mains adaptor for long periods of time.

• GUARANTEE

We guarantee your monitor against manufacturing defects for 2 years from the date of purchase on production of a valid receipt. In the first instance of a manufacturing defect, we recommend that you return your monitor to its place of purchase. If this does not satisfactorily resolve your problem, please refer to www.tomy.eu for further advice. Full terms and conditions can be found on our website: www.tomy.eu

• BATTERY SAFETY

Removal and replacement of batteries should be carried out by an adult or under adult supervision. Take special care to ensure batteries are inserted correctly, observing (+) and (-) polarity marks on battery and product. Do not short circuit the contacts in the battery compartment or the battery terminals. Remove exhausted batteries from product. Do not mix old (used) and new batteries or batteries of different types, e.g. rechargeable and alkaline or batteries of different makes/brands. Dispose of exhausted batteries safely, and never dispose of batteries in a fire. Remove batteries from product after use or if storing for long periods. Only use batteries of the same or equivalent type to those recommended.

Removal and recharging of rechargeable batteries should be carried out by an adult or under adult supervision. If you are using rechargeable batteries within the Pad Controller, they must be removed from the monitor before recharging. Do not attempt to recharge non-rechargeable batteries.

Help the environment by disposing of your product and packaging responsibly. The wheellie bin symbols indicate the product and batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please use designated collection points or recycling facilities when disposing of the item or batteries.

PROBLEM	POSSIBLE CAUSE	SOLUTION	PAGE
False Alarm (continual beeping).	Baby was removed from the cot/Moses basket without switching the Pad Controller off.	Turn the Pad Controller off.	2
	Movement Sensor Pad cord is not connected properly to the Pad Controller.	Unplug and reconnect the cord and test the function by tapping the Pad's surface	3
	Sensor Pad is not in full contact with the cot mattress, or the cot mattress is not resting on a completely flat or rigid surface.	Make sure there is no bedding between the mattress and Sensor Pad. Sensor Pad must be on a completely flat and rigid surface.	2
	Baby has moved away from Sensor Pad to the edge of the cot.	Check the positioning of the Sensor Pad as described within the Set-Up section of this booklet.	2
No Alarm (when baby is removed from the crib).	Pad Controller is detecting movement when cot is being touched.	Avoid contact with the cot when using this monitor.	4
	Pad Controller is detecting movement from outside of the cot such as motorised devices, strong draughts, central heating, etc.	Remove any nearby motorised devices (i.e. cot side mobiles/nightlights etc).	4
		Position the cot near a solid supporting wall to prevent the Sensor Pad from detecting movement from outside the cot.	4
No Movement Clicks	The click volume has been set to mute.	Increase the volume by turning the thumb wheel in a clockwise direction.	3
Random beeps	The low battery alarm will emit a 4 beep alarm every 60 seconds.	Change batteries.	3

Connecting to a Tomy TD or TDV Series Monitor

PROBLEM	POSSIBLE CAUSE	SOLUTION	PAGE
I have batteries within my TD series Monitor baby unit but my Pad Controller doesn't work	The Pad Controller requires its own battery supply unless the connected Baby Unit is powered by mains (AC) power.	Power your TD series monitor via mains.	3
		We recommend that your Pad Controller is always used with batteries as a backup in the event of a mains power failure.	4
Batteries within my Baby Unit are fine but my Pad Controller continues to alarm for low battery.	The Pad Controller will continue to draw power from its own battery supply unless the connected Baby Unit is powered by Mains (AC) power.		
How can I be assured if my connected Digital Monitor fails (dead batteries/out-of-range etc.) that my baby will be Ok?	Should a connection between the Pad Monitor and your connected Digital Monitor fail, all Movement Sensor Pad monitoring shall be transferred back to the Pad Controller.	We recommend that your Pad Controller is always used with batteries as a backup in the event of a mains power failure.	4

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