Schläfen- und Ohrthermometer
Temple and ear thermometer
Thermomètre temporel et auriculaire
Slaap- en oorthermometer
Termómetro de sienes y oídos

CE 0197

Modell / Model
Modèle / Model
Modelo: TS 04

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www.ibp-info.de

Hersteller:

AViTA Corporation, No.200.205 Provincial Free way,
Song Ling Town, Wu Jiang Su, P.R.C.

Temple and ear thermometer

CE 0197

Model: TS 04

Instruction manual

Dear Customer,

before using the thermometer, please read this entire instruction manual carefully in order to avoid causing damage as a result of using it inappropriately. Please pay particular attention to the safety information. If you pass the device on to a third party, this instruction manual must be handed over at the same time.

TABLE OF CONTENTS

Introduction.....	17
Safety information.....	18-19
Components.....	20
Display / icons.....	20-21
Accessories.....	21
Product information.....	21
Battery installation / replacement.....	22
Operating instructions.....	23-25
Storage.....	25
Technical data.....	26
Cleaning and care.....	26
Trouble shooting.....	27
Waste disposal and care of the environment.....	27
Service and Warranty.....	28

INTRODUCTION

This device conforms to the requirements of EU directive 93/42/EEC for Medical Products and has been issued with the CE 0197" conformity sign.

The ibp company confirms conformity with these requirements for the device supplied together with this instruction manual. The conformity declaration is available.

Congratulations on purchasing our temple and ear thermometer. This thermometer will allow you to take temperature readings quickly and accurately.

- | | |
|----------------|---|
| Instant | <ul style="list-style-type: none">• No waiting time• Only 0.1 seconds per reading |
| Comfortable | <ul style="list-style-type: none">• Its special ergonomic design makes it easy to use. The new MultiScan sensor can be cleaned with diluted alcohol and simplifies the thermometer's use. |
| Compact design | <ul style="list-style-type: none">• The lightweight, compact design makes it ideal for use at home or while travelling. |
| Energy-saving | <ul style="list-style-type: none">• Disconnects automatically after one minute, assuring a longer battery life and allowing a minimum of 5.000 readings. |
| Resistant | <ul style="list-style-type: none">• The extraordinary infrared technology assures greater product durability. |
| Accuracy | <ul style="list-style-type: none">• Deviation of $\pm 0.2^{\circ}\text{C}$. |
| Easy to use | <ul style="list-style-type: none">• 10 readings can be stored each for 2 persons, i.e. a total of 20 saved readings. LCD display for easy reading. |
| Flexible | <ul style="list-style-type: none">• The thermometer allows you to take readings in Celsius or Fahrenheit. |

Note:

According to extensive clinical studies, readings taken at the ear are comparable to readings taken orally by conventional thermometers.


SAFETY INFORMATION

- Only the sensor is water resistant, the device itself is not water resistant.
- Avoid touching the device on the sensor tip, thus ensuring that measurement accuracy is not affected by heating and soiling of the sensor casing.
- The thermometer is only to be used for measuring the temperature of the human body.
- Incorrect operation and improper handling may cause problems with the appliance and/or injury to the user.
- The unit may be used only for its intended purpose. No liability is accepted for any damage arising from improper use or faulty handling.
- Never attempt to repair the appliance yourself.
- Never attempt to open the housing.
- Do not introduce objects of any kind into the housing.
- Do not operate the unit if the unit has been dropped or otherwise damaged. In such cases, send the unit to a service centre for testing and any necessary repairs.
- This unit is not suitable for commercial use.
- Children cannot recognise the dangers associated with improper handling of electrical devices. You should therefore never allow children to use electrical appliances except under supervision.
- Do not throw batteries away with your household rubbish!
Please dispose of batteries in accordance with the law!
Batteries are hazardous waste!
Never short-circuit a battery!
Do not throw the battery into the fire!
Keep batteries away from children!
- The appliance contains high-precision components. You should keep it away from extremes of temperature, humidity and direct sunlight. Avoid dropping or knocking the unit and protect it from dust.
- The temple and ear thermometer is not suitable for rectal measurements.

- The accuracy of measurements may be limited by the following factors: increased sweat formation on the forehead, the intake of vasoconstrictive medication and skin irritation. The temple and ear thermometer is calibrated for ear temperature measurement at the manufacturer, inside the ear the sensor is protected against external temperature and consequently produces a more precise measurement. On the temple or forehead the skin is cooler, depending upon ambient temperature, i.e. the measurement displays 0.5 0.6 °C less than inside the ear. In an extremely cool environment this deviation may be even greater.
- Please remember that this does not replace a visit to your doctor.
- Self-diagnosis and self-treatment using this measurement device are dangerous. Only a doctor can evaluate your measurements. It is important that you follow the instructions of your doctor.
- Clean the water resistant sensor after use. Read the chapter on cleaning and care for more details.

COMPONENTS

- A) Water resistant sensor
- B) Battery cover
- C) Memory button
- D) Power & Scan button
- E) LCD display

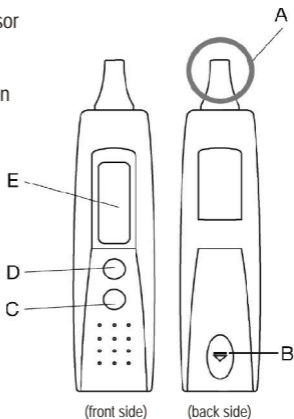DISPLAY / ICONS

888.8 Temperature in degrees

°C Degrees Celsius

°F Degrees Fahrenheit

←||||| In the process of reading temperature

😊 😞 Reading completed

 Memory symbol and number of reading

 Replace battery

 Happy face to indicate temperatures below
38° Celsius

 Sad face to indicate temperatures above
38° Celsius

ACCESSORIES

Instruction manual, Temple and ear thermometer,
Battery type CR-2032, Storage box


PRODUCT INFORMATION

The temple and ear thermometer is a delicate electronic device designed for the reading of body temperature. It allows accurate and reliable readings by reception of the infrared rays in the ear or at the temple. The thermometer converts the infrared energy measured into a temperature value. However, it should be noted that the body's temperature varies depending upon age and the time that the reading is taken. This device may also be used to measure other types of temperature, for instance, of the skin, water, milk and ambient temperature. It is important to point out that it is not necessary for the thermometer to be in direct contact with the material. The optimum distance between the object and the thermometer is approximately 0.2- 0.39 inch (0.5-1.0 cm).

BATTERY INSTALLATION / REPLACEMENT

Please only replace the battery when the weak battery symbol appears in the LCD display (E) (see page 21).

- Slide back the battery cover (B).
- Lever out the old battery with a suitable tool (e.g. a small slotted screwdriver).
- Please use a type CR-2032 button cell battery.
- Insert the battery with the + terminal facing upwards.
- Slide the battery cover (B) back into position.
- Please dispose of old batteries at the foreseen collection points. Read the chapter on waste disposal and care of the environment for more details.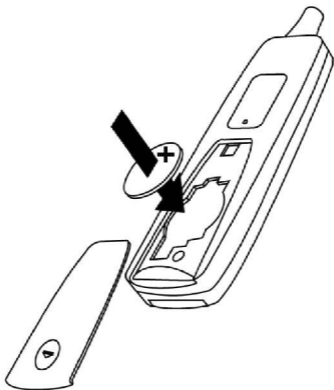


OPERATING INSTRUCTIONS

The meanings of the symbols can be found on pages 20 and 21 of this instruction manual.

Switching on the temple and ear thermometer:

- Press the On/Off button (D) for one second, the device is switched on following calibration.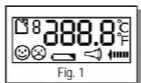

- After calibration "M1" or "M2" (memory position 1 or memory position 2) appears on the display (fig.2). Press the On/Off button (D), if you wish to move from one memory position to the other. The most recent measurement is displayed after 2 seconds.



Switching off the temple and ear thermometer:

- Hold down the On/Off button (D) for approximately 5 seconds. The device is switched off automatically after one minute.

Measurements in the ear (most accurate measurement):

- Introduce the sensor (A) deep into the ear. The sensor (A) must be positioned in the ear canal in order to record the sustained heat of the eardrum (See fig. 3).
- Press the On/Off button (D) briefly. You will hear an acoustic signal. Remove the temple and ear thermometer in order to read the results on the display (fig.4).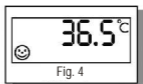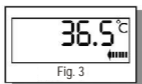

Measurements on the temple/forehead:



Please note that the ambient temperature outside of the ear may influence the measurement result. The cooler the ambient temperature, the cooler the skin is, i.e. the measurement may be 0.5-0.6 °C below that of the temperature in the ear. Alternatively, you can also take a measurement from the temple or forehead.

- Hold the sensor (A) directly on the temple. Press the On/Off button (D) briefly. You will hear an acoustic signal. This confirms the measurement. You can now read the result from the LCD display (E).

Changing the display from °C / °F:



This temple and ear thermometer can be altered from Celsius to Fahrenheit.

- Hold down the On/Off button (D) and press the save button (C) briefly. (See fig. 5.1 and 5.2)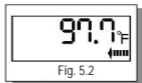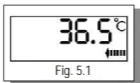



Meaning of the face symbols:

In the case of a result below 38 degrees Celsius a smiling face will appear on the LCD display (E), in contrast, a sad face is displayed at temperatures above 38 degrees, with 2 acoustic signals also heard. (See fig. 6 and 7)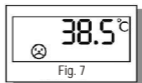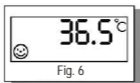

Error message on the LCD display:

The measurement temperature of the device lies between 10° and 50° Celsius. In the event of a result below 10 degrees «Lo» appears, at over 50 degrees «Hi» is displayed (see fig. 8 and 9).

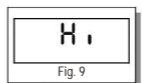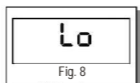Saving measurements:

The temple and ear thermometer automatically saves the last 10 measurements, numbered from 0 to 9. The most recent measurement is shown in the upper left as 0, the oldest measurement as 9. (See fig.10)

- To access the latest measurements press the memory button (C).
- Repeated pressing of the memory button (C) enables the values from 0 to 9 to be accessed individually, one after another.
- It is advisable to wait at least 5 seconds between individual measurements in order to ensure measuring accuracy.

STORAGE

Store the device in the box provided after use.

TECHNICAL DATA

Model:	TS 04
Automatic switch-off:	after approx 1 minute
Measurement range:	10.0°C- 50.0°C (50° F- 122°F)
Operating temperature:	16.0°C- 40.0°C
Storage/transport Temperature:	-20.0°C-50.0°C, humidity 75% RH min.
Laboratory accuracy compared with test device measured in ear:	± 0.2°C between 36°C - 39°C ± 0.3°C at below 36°C or over 39°C
Battery:	1x 3 V. battery, type CR-2032
Weight including battery:	52 g
Dimensions:	130 x 33 x 23 mm
Explanation of symbols:	

Warning:
follow instructions for use.

Device classification: type BF

CE mark of conformity

CLEANING AND CARE

Never use harsh cleaning fluids or place the temple and ear thermometer into water or other fluids.

Cleaning of casing:

The casing of the temple and ear thermometer is not water resistant, please use a dry cloth for this.

Cleaning of sensor:

Please clean the water resistant sensor with a soft, damp cotton cloth or cotton wool buds. An alcohol solution may also be used.

TROUBLE SHOOTING

Permanent display of insufficient temperature

- The sensor is not inserted far enough into the ear canal. The tip must be entered deeply, otherwise measurements may be too low.
- The sensor is dirty. Clean the sensor (A) with a soft damp cloth or cotton wool buds with alcohol solution.

The result is outside of the measurement range

- Switch the device on prior to inserting it in the ear or holding it to the temple. It is possible that the measured temperature lies outside of the range when you begin the measurement, before the sensor reaches the eardrum or temple. Please repeat the measurement.

Weak battery

- If the weak battery signal appears, please replace the button cell battery (see battery replacement).

WASTE DISPOSAL AND CARE OF THE ENVIRONMENT

Do not throw away the used battery with your household rubbish but put it with the hazardous waste or in a battery collection box at a specialist shop.

At the end of its life cycle, this product must not be disposed of in regular domestic waste, but must be taken to a collection site for recycling electric and electronic devices (disposal of used appliances).

If you have any questions on disposal, our Service Department or your dealer will be pleased to help you.

SERVICE AND WARRANTY

Service centre addresses:

England:

International Business Products Limited
6 Newfield Court, West Houghton, Bolton, BL5 3 SA
England, Tel.: +44 (0)1942 819 695
www.ibphealthcare.com

India:

sano health care products pvt. Ltd.
Corporate Office: 28 Landsdown Terrace, 1st Floor,
Kolkata 700 026, Tel.: +91 33 2465 0383-85

Warranty and compensation for damage:

If your appliance is non-operational due to a manufacturing or material defect, we will carry out repairs free of charge for 24 months from the date of purchase (date on proof of purchase or receipt).

Batteries are excluded from the warranty.

The warranty excludes damage resulting from non-compliance or only partial compliance with these instructions for use.

Any further claims such as compensation for damage are likewise excluded.

Please send your appliance to our service centre address together with your proof of purchase or receipt and a description of the complaint. Please also attach the warranty documents if you would like to file a claim under warranty.

Manufactured by:

AVITA Corporation, No.200.205 Provincial Free way, Song Ling Town, Wu Jiang Su, P.R.C.

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ibp innovative business promotion GmbH
AViTA Corporation, No.200.205 Provincial Free way, Song Ling Town, Wu Jiang Su, P.R.C.