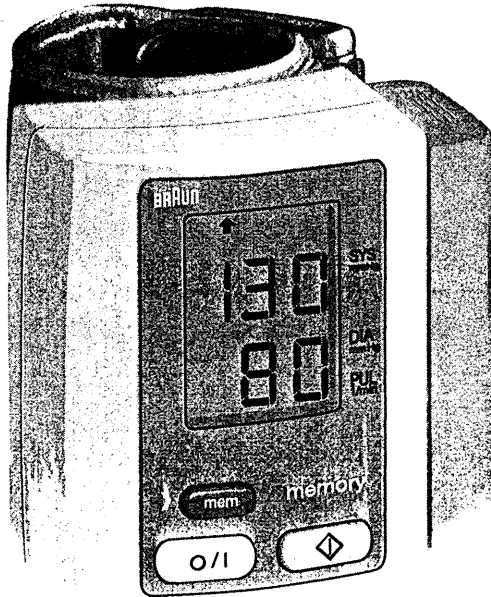
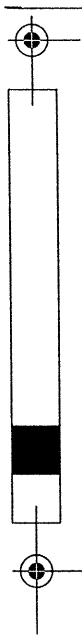
BRAUN

BP 1000 BP 1500

Type 6050
Type 6052

English

Français

Español

Thank you for purchasing a Braun product.
We hope you are completely satisfied with
your new Braun VitalScan.

If you have any questions, please call:

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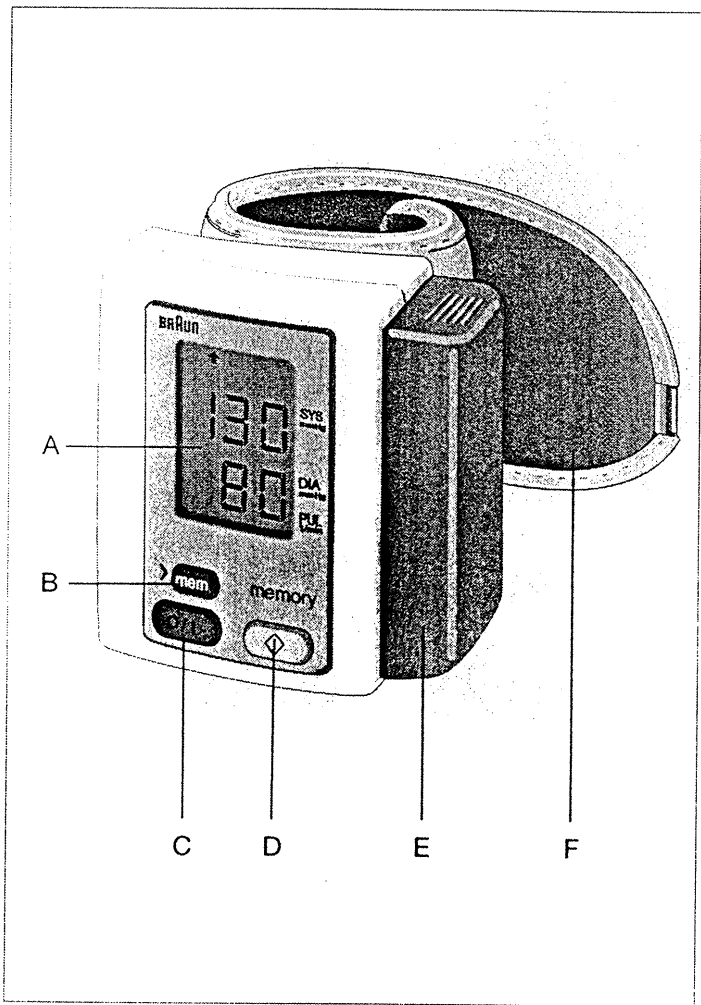
www.braun.com

English

Braun VitalScan™ is a blood pressure monitor that has been carefully developed for easy and accurate blood pressure measurement at the wrist.

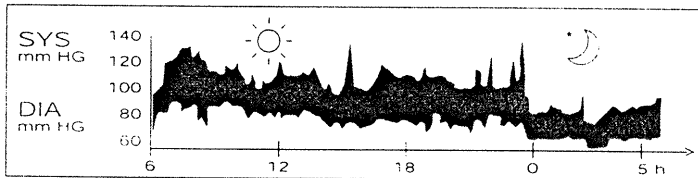
The measuring accuracy has been tested at the time of manufacture and proven by clinical tests.

However, to ensure accurate readings, carefully read these use instructions.

What you should know about blood pressure

Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. That is why readings can fluctuate when taken at different times of the day.

Blood pressure measured in a doctor's office provides only a momentary value. Repeated measurements at home better reflect one's actual blood pressure values.

Many people may have a different blood pressure when they measure themselves at home, because they tend to be more relaxed than when in the doctor's office.

Regular blood pressure measurements taken at home can provide your doctor with valuable information on your actual «everyday» values.

The World Health Organisation (WHO) and the American Heart Association (AHA) have set up the following standard blood pressure values:

Blood pressure (mm Hg)	Normal values	Mild hypertension	Moderate and severe hypertension
SYS = systolic = upper value	up to 140	140 – 180	over 180
DIA = diastolic = lower value	up to 90	90 – 105	over 105

Important

This product is intended for household use only.
Keep product and batteries away from children.

People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since different values may be measured in such cases. There may be a difference between values measured at the wrist and those measured at the upper arm.

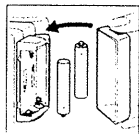
If you are under medical treatment or receiving medication, please consult your doctor in case of any doubt.

The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

Product description

- A Display (LCD) [↑ ready/inflation symbol, ↓ deflation symbol]
- B Memory button (only with BP 1500 model)
- C On/off button «O/I»
- D Start button «»
- E Battery compartment
- F Cuff

Inserting/replacing batteries



- Use 2 «AA» type, alkaline batteries (e.g. Duracell) as supplied with this product.
- Open the battery compartment cover and install two batteries with correct polarity (see printing in battery compartment).
- New alkaline batteries will provide approximately 300 measurements.

Key rules for accurate blood pressure measurement


- Always take readings at the same time of day, ideally in the morning, under the same conditions.
- Don't measure within 30 minutes after smoking, coffee or tea consumption.
- While taking a measurement, sit down, relax, keep still, do not move or speak.
- Hold the product at heart level. For your convenience, make use of the carrying case as an armrest.
- Always wait about 3 minutes before repeating a measurement.

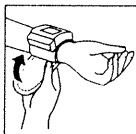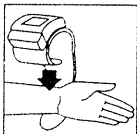4 easy steps to measure blood pressure


1 Attach the cuff

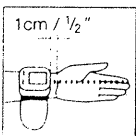- Take off your watch and any jewelry. Wrap the cuff around your wrist.
- Place the product on the palm side of your wrist, roughly 1/2 inch (1 cm) below the palm line. Fasten the cuff snugly. (Cuff is suitable for wrist circumference of 5.3 - 7.7 inches / 13.5 - 19.5 cm.)


2 Hold the product at heart level

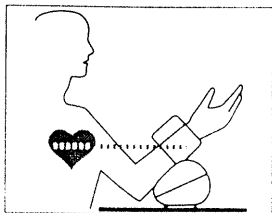- Place the hand with the palm up and lightly open.
- Hold the wrist at heart level by resting your forearm on the carrying case. The wrist must be held at heart level for an accurate reading. Holding the wrist above or below heart level may lead to inaccurate readings.



3 Switch on

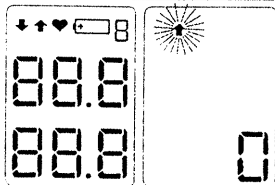
- Press the on/off button.
All display segments appear.
- The product is ready for use as soon as the buzzer sounds three times and «0» appears on the display.

4 Start the measurement

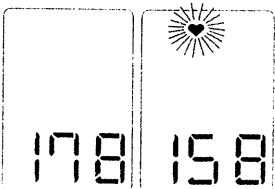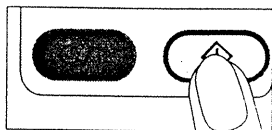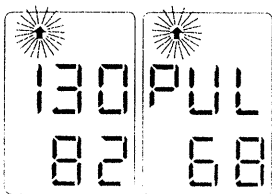- Press the start button.
- The cuff inflates automatically up to a pressure above your systolic value. If you have high blood pressure, the product will repressurize until the appropriate upper inflation value is reached. Then, the cuff is deflated at a constant rate. While measurement is in progress, do not move your wrist or fingers, nor tighten your arm or hand muscles. When a pulse is detected, the buzzer sounds and the «♥» symbol starts to flash.


- At the end of measurement, the buzzer sounds for about 1 second, the cuff is deflated completely and both blood pressure readings are displayed alternately with the pulse rate.

SYS = systolic = upper value and
 DIA = diastolic = lower value
 PUL = pulse rate


- To switch off, press the on/off button. If not turned off manually, the product automatically switches off after approximately 3 minutes.

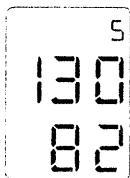Pressure Track™/memory function (BP 1500 only)

The 7 last blood pressure measurements are memorized automatically.

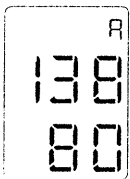Up to 7 stored values can be recalled by repeatedly pressing the «mem» button. The number appearing in the top right corner of the display, indicates the current memory position.

Average function

If pressing the «mem» button again after reaching the last memory position, the display shows the average value of all stored measurements with the letter «A» appearing in the top right corner of the display. Changing or removing batteries will cause all readings in the memory to be lost.

What to do if...

Problem	Situation	Solution
LCD shows «Err330»	Inflation above 330 mm Hg	Repeat measurement.
LCD shows «Err»	Error due to body movement during measurement. Product has been wrongly positioned.	Repeat measurement.
 No display	Batteries are weak or empty. No batteries inserted or polarity of batteries is wrong.	Replace batteries. Insert new batteries or just check polarity.
No acoustic signals (beeps)	Cuff is not properly fitted.	Fit cuff again and fasten snugly.
Cuff cannot be pressurized.	Product is switched off. Cuff is too loose.	Switch product on. Fasten cuff snugly.
Readings are too high or too low (improbable values).	Product was not held at heart level (deviation: approx. 8 mm Hg per 10 cm). Cuff was not properly fitted. Body movement during measurement.	Repeat measurement with correct arm posture.

If any of these errors persists, have the product checked at a Braun Service Center. If you have further questions, call Braun Info-Line: 1-888-848-2572 (1-888-VITALSCan).

Care and cleaning

Clean the housing of the product with a damp cloth. Stains on the cuff can be removed with a damp cloth and soapy water. Do not wash or dry-clean the cuff. Do not store the product at a place exposed to direct sunlight or high humidity.

Calibration

This product is initially calibrated at the time of manufacture. If used according to the use instructions, periodic re-calibration is not required. If at any time you question the measuring accuracy, please contact Braun at 1-888-848-2572 or 1-888-VITALSCan.

Specifications

Model:	BP 1000 / BP 1500
Operating principle:	Oscillometric method
Indicator:	7-digit LCD display
Measuring range:	0 to 300 mm Hg (cuff pressure) 40 to 160 pulses/min. (pulse rate)
Accuracy:	± 3 mm Hg (cuff pressure) ± 5 % of reading (pulse rate)
Inflation:	Automatic inflation (air pump)
Deflation:	Electronic control valve
Power supply:	2 «AA» type batteries (LR 6)
Operating temperature:	+ 10 to + 40 °C (50 to 104 °F)
Storage temperature:	- 20 to + 60 °C (- 4 to 140 °F)
Humidity:	85 % RH or below
Cuff:	Hard cuff, wrist circumference 5.3 - 7.7" or 13.5 - 19.5 cm

Type BF equipment

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or automated sphygmomanometers.

Subject to change without notice.

Manufactured for Braun
Made in Japan

Limited 3-year warranty

Braun warrants this product against any defects that are due to faulty material or workmanships for a period of three years from the original date of consumer purchase. This warranty applies when used for normal household use in accordance with the use instructions and excludes the battery and damage to the product resulting from accident or misuse. This product is not warranted when used in a professional environment.

In no event shall Braun be liable for any special, incidental, indirect or consequential damages in connection with the purchase or use of this product or costs over the original cost of the product.

If the product should become defective within the warranty period, contact Braun's Technical Support Center at 1-888-848-2572 for repair or replacement at no charge. Braun reserves the right to replace a defective product with the most comparable product currently available.

While this warranty gives you specific legal rights, you may also have other rights which vary from state to state.

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