BRAUN

BP 2590

PrecisionSensor pro

Type 6 059

Braun Infoline

English 4, 58, 62

Français 13, 58, 62

Español 22, 59, 62

Português 31, 60, 62

Italiano 40, 60, 62

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
www.braun-medical.com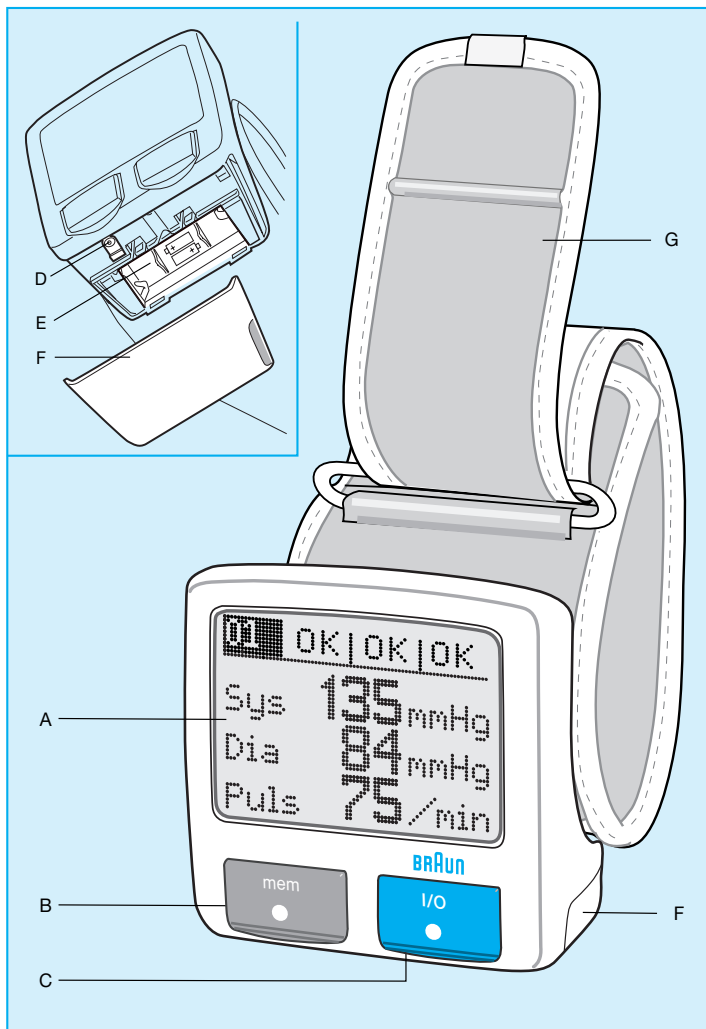

English

Braun PrecisionSensor pro is a blood pressure monitor that has been developed for easy and accurate blood pressure measurements at the wrist. The accuracy of the product has been tested at the time of manufacture. Measurements can be saved.

For reliable measurements, blood pressure monitors should always be positioned at an appropriate measuring position at heart level.

Therefore the Braun PrecisionSensor pro has an active positioning system that guides the patient's arm via arrows on the display to the appropriate measuring position at heart level.

However, to ensure accurate results, carefully read these instructions.

Hypertension in the medical practice¹⁾

Hypertension is very critical as risk factor for a number of cardiovascular related diseases. Thus, it is one of the main risk factors for stroke, coronary heart disease, cardiac insufficiency and renal disease.

The World Health Organization (WHO) estimates that up to 20 % of the world's adult populations has high blood pressure. The prevalence is even higher in the older population segments resulting in up to 50 % of people affected by hypertension.

These data reveal that blood pressure measurement plays a central role in the medical practice either as screening method for undiagnosed individuals or as treatment monitoring tool for diagnosed patients. Blood pressure measurement in the practice represents the foundation for diagnosis and therapy of hypertension apart from the widespread use of patient home monitoring and special diagnostic procedures, such as Ambulatory Blood Pressure Measurement (24-hour-measurement).

The importance of the blood pressure measurement in the practice's environment calls for a high performance standard with regard to quality and reliability of the professional measuring device. Braun PrecisionSensor pro has been designed to meet these demanding requirements. The device features a newly developed electronic positioning sensor that commands an active positioning system enabling the user to reliably find the appropriate measuring position at heart level. In this way accurate and reproducible

readings are achieved. Moreover, the device monitors the appropriate positioning at definite intervals during the measurement cycle, thus providing additional safety and reliability. This watchdog feature allows patient self measurement in the practice, which may be beneficial in reducing the problem of white coat hypertension.

With its technical design, Braun PrecisionSensor pro is the ideal supplementary device for the medical practice.

For further information on blood pressure measurement or topic information on diagnosis and therapy of hypertension you may visit the Braun professional home page at www.braun-medical.com.


Important

- Deviations in blood pressure values may occur with people suffering from cardiac arrhythmia, angiostenosis, arteriosclerosis in the upper extremities, diabetes or with users of cardiac pacemakers. In case of doubt, please carry out a comparative measurement with the stethoscope method.
- The cuff of this product is suitable for a wrist circumference between 13.5 and 20 cm. Measuring deviations may occur if patient's wrist is not within this range.
- Patient's self-measurement in the medical practice: Prior to the first self-measurement, it is necessary to instruct the patient how to use the appliance correctly.

Product description

- A Display (LCD)
- B Memory button
- C I/O (on/off) button
- D Select button
- E Battery compartment
- F Battery compartment cover
- G Cuff

Inserting batteries

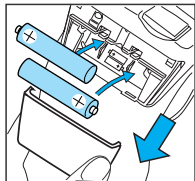- Use only alkaline batteries of the type LR 03 (AAA) (e.g. Duracell-Ultra) as supplied with this product.
- Remove the battery cover (F) and insert two batteries with correct polarity (see symbol in the battery compartment (E)).
- New batteries (Duracell-Ultra) will provide approximately 300 measurements.
- Only discard of empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your dealer.
- Repeat a measurement no sooner than after 3 minutes.


Key rules for accurate blood pressure measurement²⁾

- Blood pressure should be measured on a patient to be seated who has had a rest of approx. 3-5 minutes. If indicated, a measurement can also be taken on a person in supine position (refer to «Measurement on a person in supine position»). During measurement, the patient should not move, nor speak.
- Prior to the measurement, the patient should avoid heavy physical exercise or psychic excitement.

- The patient should avoid drinking large quantities before the measurement; the bladder should have been emptied as far as possible.
- Avoid taking measurements within 60 minutes after the patient has consumed alcohol or nicotine.
- If a patient takes blood pressure effective medication, ask when he/she last took it.
- Wait approximately 3 minutes before repeating a measurement.

Measuring


For every blood pressure measurement at the wrist an appropriate measuring position at heart level is essential for accurate results.

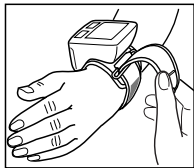
1. Fasten the appliance on the patient's left arm on the radial side of the forearm (see illustration). Make sure the cuff (G) fits tight. Stand close to the patient in a position so that you can easily see the display and the control buttons. Press the «I/O» button (C). An arrow appears on the right hand side of the display (A). If there are readings stored in the memory, «mem» appears in the lower left corner of the display.

2. Help the patient to take the measuring position as shown. The arms should remain close to the body. Make sure the appliance is positioned approximately at the person's heart level. Move the forearm of the patient up or down (see illustration) in the direction of the arrow.

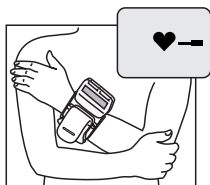
3. When a measuring position has been reached, you will hear an acoustic signal and the display shows a heart symbol. Both indicate that the appropriate measuring position has been reached. The measurement now starts automatically.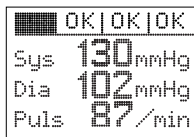

4. When the measurement is complete, an acoustic signal will sound again, and the reading is displayed.

Upper (systolic) value
Lower (diastolic) value
Pulse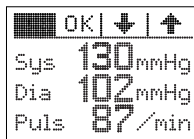

Additionally, on the upper display section, three status boxes will appear to confirm that the appropriate measuring position was kept during the complete measuring process.

All three boxes have to show «OK». If the following symbols («↓» = too high; «↑» = too low) appear, the appropriate measuring position was not maintained and the measurement has to be repeated.

Important

15 seconds after switching on the device, the measuring process will begin automatically, even if the appropriate measuring position has not been reached. If at any time you want to discontinue the measurement, press the I/O button.

Memory


The blood pressure monitor can store up to 30 readings along with the status box information. Readings are stored automatically after each measurement.

To recall stored readings, first switch off the device. Then press the memory button (B). Repeatedly pressing the memory button will call up one by one all stored readings (including status box information). The number in the upper left corner of the display indicates the memory position (01 being the oldest, 30 the most recent).

Holding down the memory button for more than a second will delete all memory data completely. After clearing the memory, the appliance will turn off automatically.

Turning off the product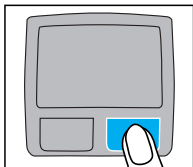


Press the I/O button to turn the product off. Otherwise the product will turn off automatically after 2 minutes.


Measurement on patients in supine position

When taking a reading on a patient in supine position the appropriate measurement position at heart level of the Braun PrecisionSensor pro device is normally achieved automatically. The active positioning system is not needed in this case. In order to guarantee a correct measurement the position sensor should be switched off for the measurement (refer to «Pre-setting the position sensor»).

Pre-setting the position sensor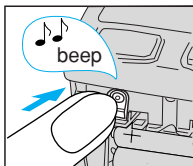

The active positioning system of the appliance allows accurate and repeatable blood pressure measurements in the practice. Therefore, the position sensor is activated in the pre-settings at the time of manufacture. We recommend that you keep it activated. In certain cases, e.g. prevailing measurements on persons lying down, it may be necessary to de-activate the position sensor. For this purpose, you will find a select button (D) underneath the battery compartment cover.

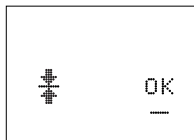First turn off the product. Keep the select button pressed, until an acoustic signal sounds. The display shows a flashing «OK» next to the position sensor symbol (see illustration).

If you wish to turn off the position sensor, press the «I/O» button (C): The display changes from «OK» (on) to «-» (off). Press the memory button to confirm the new setting. Then the product turns off automatically. In order to re-activate the position sensor, proceed as described above.

Storage, cleaning and disinfection of the product

Cleaning and disinfection of the device housing and the cuff may be done with commercially available cleaning and disinfection solutions. Preferably undyed solutions should be used in order to avoid discolouring of the device housing. To clean wipe cuff and housing with a cotton swab slightly moistened with the solution and immediately wipe dry with a clean cotton swab.

Do not store the product in a place exposed to direct sunlight or high humidity.

Calibration

This product has been calibrated at the time of manufacture. If used according to the use instructions, periodic re-calibration is not required.

Manufacturing date is given by the LOT number on the back of the product. The first number after LOT represents the last digit of the year of manufacture. The next three digits give the day of the year of manufacture.

An example: LOT 0116 xx xx.... this product has been manufactured on the 116th day of the year 2000.

What to do if ...

Problem

Situation

Batteries are weak.
Measurements still possible.

Solution

Battery replacement required soon.

Batteries are empty.
Memory readings can be called up but measurements not possible.

Replace batteries.

Cuff is too loose or cannot be pressurised.

Refit the cuff tightly.

Err 85

Error message with a number higher than 64

Refit cuff tightly, take new measurement.

If an error message with a number lower than 64 appears, or if any other errors persist, have the product checked at a Braun Service Centre. If you have further questions, call the Braun Infoline.

Specifications

Model:	BP 2590
Operating principle:	Oscillometric method
Indicator:	Liquid Crystal Display
Measuring range:	0 - 300 mm Hg (cuff pressure) 40 - 160 / min (pulse rate)
Blood pressure measurement:	50 mm Hg (minimal diastolic value) 250 mm Hg (maximum systolic value)
Accuracy:	± 3 mm Hg (cuff pressure) $\pm 10\%$ (pulse rate)
Inflation:	Automatic
Deflation:	Electronic control valve
Power supply:	2 alkaline batteries type LR 03 (AAA) (e.g. Duracell-Ultra)
Operating temperature:	+ 10 °C - + 40 °C
Storage temperature:	- 20 °C - + 60 °C
Humidity:	85% RH or below
Cuff:	Suitable for wrist circumference 13.5 to 20 cm

Type BF equipment

Subject to change without notice.

This product conforms to the provisions of the EC directive 93/42/EEC (Medical Device Directive).

- 1) WHO (1998); The World Health Report 1998; Life in the 21st century; WHO Geneva
- 2) Recommendations for blood pressure measurements; Deutsche Liga zur Bekämpfung des hohen Blutdrucks e.V.; Deutsche Hypertonie Gesellschaft; Heidelberg

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English

Guarantee

This product is guaranteed for 1 year from date of purchase against material and/or workmanship defects. These will be eliminated either by repairing or exchanging the appliance as we may choose. All other claims, including for damages, are not covered by this guarantee. Service under the guarantee does not affect its expiry date.

Claims only valid in countries where product is officially sold and if accompanied by a stamped and dated guarantee card.

Claims arising out of the sales contract with the vendor are not affected by this guarantee.

This guarantee in no way affects your rights under statutory law.

For UK only:

Braun Gold Seal Service

To maintain your appliance at peak performance it is strongly recommended that you have it checked every two years by an approved Gold Seal Service Centre.

In the unlikely event that you experience a problem with this product, Braun (UK) Ltd. promise to resolve the problem within 5 days (excluding weekends) of receipt by an approved Gold Seal Service Centre.

UK including Northern Ireland, Channel Islands and Isle of Man:

To contact your nearest Gold Seal Service Centre please telephone this number: 0870 514 3223.

Ireland: Gavins Electronics Ltd.
83 Lower Camden St.
Dublin
Call Dublin (01) 4 75 71 09

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Australia

Gillette Australia Pty. Ltd.,
Private Bag 10, Scoresby,
Melbourne, Victoria 3179,
☎ 3 9757 4704

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Ets. André Haan S.A.,
B.P. 335, 97161 Pointe-à-Pitre,
☎ 26 68 48

Hong Kong

Audio Supplies Company,
Room 506,
St. George's Building,
2 ICE House Street,
Hong Kong, ☎ 25 24-93 77

Hungary

Kisgep KFT,
Pannonia u. 23,
1136 Budapest,
☎ 1 349 49 55

Iceland

Verzlunin Pfaff h.f.,
Grensasvegur 13,
Box 714, 121 Reykjavik,
☎ 5 33 22 22

India

Braun Division,
c/o Gillette Div. Op. Pvt. Ltd.,
34, Okhla Industrial Estate,
New Delhi 110 020,
☎ 11 68 30 218

Ireland (Republic of)

Gavins Shaver Centre,
83/84, Lower Camden Street,
Dublin 2,
☎ 47571 09

Israel

S. Schestowitz Ltd.,
6 Josef Caro St., Tel-Aviv, 67014
☎ 1 800 335 959

Italia

Servizio Consumatori Braun
c/o Gillette Group Italy S.p.A.,
Via G.B. Pirelli, 18,
20124 Milano,
☎ 02 / 667861

Jordan

A. Dajani & Sons Co.,
Shamaisani/Sherif Jamil Bin
Naser Street, P.O. Box 926976,
Amman 11110,
☎ 6552 5545

Kenya

Radbone-Clark Power
Technics Ltd.,
P.O. Box 49197,
Mombasa Road, Nairobi,
☎ 2 82 36 60

Korea

STC Industries
Corp. Braun Division,
32, Mullaee-Dong 3-KA,
Youngdeungpo-Ku,
Seoul 150-090,
☎ 22 678 97 32

Kuwait

Union Trading Co. W.L.L.,
P.O. Box 28,
Safat Code 13001, Kuwait,
☎ 4 83 46 05

Latvia

Latintertehserviss Co.,
72 Bullu Street, House 2,
Riga 1067,
☎ 2 40 37 19

Lebanon

Aurora s.a.r.l./Fital Group,
P.O. Box 110-773,
Beirut, ☎ 1 51 20 02

Lituania

Elektronas AB,
Joint Stock Company,
Kareiviu 6,
2600 Vilnius,
☎ 277 76 17

Luxembourg

Sogel S.A.,
1 Dernier Sol, BP 1941,
1019 Bonnevoie,
☎ 4 00 50 51

Libya

Ariej for Domestic & Electrical
Appliances,
First September street,
P.O. Box 4996, Tripoli,
☎ 21 333 8492

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Philippines

Gillette Philippines Inc.,
Sucat, Paranaque City,
Villongco Road,
☎ 2 00 22 to 29

Poland

Gillette Poland S.A.,
Budynek Orion, I pietro,
ul. Domaniewska 41,
02-672 Warszawa,
☎ 548 88 88

Portugal

Gillette Portuguesa, Ltd.,
Braun Service,
Centro Empresarial
Torres da Lisboa,
Rua Tomás da Fonseca,
Torre G-9^oB,
1600 Lisboa, ☎ 17 22 00 63

Qatar

Al-Baker Trading Co.,
P.O. Box 9, Doha, ☎ 621 059

Réunion

Dindar Confort,
58, rue Maréchal Leclerc,
P.O. Box 12, 97400 St. Denis,
☎ 40 12 50

Romania

Tamisa Trading SRL,
Soseaua Pipera - Tunari 2C,
Comuna Voluntari 729941,
Judetul Ilfov,
☎ 1 49 083 60

Russia

Braun Russia
c/o Gillette Int.,
Paveletskaja sq. 2, Bldg. 3,
8 Floor,
113054 Moscow,
☎ (095) 258 62 65

Saudi Arabia

Salem M. Bakhshwain
Sons Co. Ltd., P.O. Box 743,
21421 Jeddah,
☎ 65 72 900

Schweiz/Suisse/Svizzera

Telion AG, Rütistrasse 26,
8952 Schlieren,
☎ 17 32 15 11

Singapore (Republic of)

Beste (S) Pte. Ltd.,
6 Tagore Drive, # 03-04 Tagore
Industrial Building,
Singapore 787623,
☎ 4 552 24 22

Slovenia

Coming d.d.,
Slovenceva 024,
1000 Ljubljana,
☎ +386-01-560-40-00

Slovakia

Techno Servis Bratislava
spol. s.r.o.,
Bajzova 11/A,
82108 Bratislava,
☎ 7 / 55568161

South Africa (Republic of)

Fixnet cc,
17B Allandale Office Park,
P.O. Box 5716,
Cnr Morkels and Le Roux Street,
Johannesburg 1685,
Midrand,
☎ 11 315 9260/1

South Pacific

Woodward's Agencies,
39 Taylor's Rd., Kingston,
Norfolk Island, ☎ 24 05

St. Maarten

Ashoka,
P.O. Box 79,
Philipsburg, Netherlands Antilles,
☎ 52 33 74

St. Thomas

Boochoand's Ltd.,
31 Main Street,
P.O. Box 5667,
00803 St. Thomas,
US Virgin Islands,
☎ 340 776 0302

Suomi

Gillette Group Finland Oy/Braun,
Niittykatu 8, PL 9, 02200 Espoo,
Asiakaspalvelu 0203 77877
Tekninen neuvonta 09-45 28 71

Sverige

Gillette Group Sverige AB,
Division Braun, Box 1056,
172 22 Sundbyberg,
☎ 020-21 33 21

Syria

Unisyria, P.O. Box 35002,
Damascus,
☎ 11 622 2881

Taiwan, R.O.C.

Audio & Electr. Supplies Ltd.,
Brothers Bldg., 10th Floor,
85 Chung Shan N Rd., Sec. 1,
Taipei (104),
☎ 2 252 33 283

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Yemen (Republic of)

Saba Stores for Trading,
26th September Street,
P.O. Box 5278,
Taiz,
☎ 4 226 6359

Yugoslavia

BG Elektronik,
Bulevar revolucije 34,
11000 Beograd,
☎ 11 3240 030

Braun BP 2590

PrecisionSensor pro

Guarantee Card
Carte de garantie
Tarjeta de garantía
Cartão de garantia
Certificato di garanzia
Κάρτα εγγυήσεως

Braun BP 2590

PrecisionSensor pro

Registration Card
Carte de contrôle
Tarjeta de registro
Cartão de registo
Cartolina di registrazione
Κάρτα καταχώρησης

Service notes

Name and full address of purchaser
Nom et adresse complète de l'acheteur
Nombre y dirección completa del comprador
Nome e morada completa do comprador
Nome e indirizzo completo dell'acquirente
Όνομα και πλήρης διεύθυνσις αγοραστή

Date of purchase
Date d'achat
Fecha de adquisición
Data de compra
Data d'acquisto
Ημερομηνία αγοράς

Date of purchase
Date d'achat
Fecha de adquisición
Data de compra
Data d'acquisto
Ημερομηνία αγοράς

Stamp and signature of dealer
Cachet et signature du commerçant
Sello y firma del proveedor
Carimbo e assinatura do revendedor
Timbro e firma del negozio
Σφραγίδα και υπογραφή καταστήματος

Stamp and signature of dealer
Cachet et signature du commerçant
Sello y firma del proveedor
Carimbo e assinatura do revendedor
Timbro e firma del negozio
Σφραγίδα και υπογραφή καταστήματος