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NASAL CARE SYSTEM 10

The Petit Terraillon nasal care 10 system is an efficient and hygienic solution for blowing the noses of children under 3.

READ BEFORE USE :

- This device is not a toy. Keep out of reach of children.
- Do not use this device other than for the intended purpose: cleaning/removing nasal secretions only.
- If you detect any abnormal behaviour in your baby after using this device, stop using it and consult your doctor immediately.
- In case of nose bleeds, stop and consult a doctor immediately.
- People who suffer from nasal complaints, allergies or ear/eye problems must consult their doctor before using this device.
- This device is suitable for children under 3 and all persons unable to blow their noses by themselves. Only adults may use this device.
- While using this device, do not hold your breath. Close your mouth or swallow saliva to avoid vacuum pressure causing an eardrum imbalance.
- This device contains batteries and accessories. Ensure they are kept out of reach of children as they can be swallowed. If any items are swallowed, consult a doctor immediately.
- The battery may explode if misused. Do not attempt to recharge the battery or take it apart and do not throw it into fire.
- Remove the batteries if you do not intend using the device for a prolonged period of time.
- Do not throw old batteries away. Place them in a container specially provided for flat batteries so that they may be collected and recycled.
- For home use only.

- Do not immerse in water or any other liquid.
- Clean the device with a damp cloth.
- The cover, top of the receptacle and receptacle can be dismantled easily and washed in water.

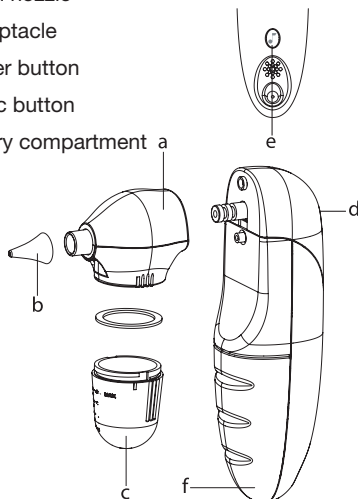
In accordance with the European directive 2002/96/CE, this product must not be disposed of with other household waste.

Dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

NASAL CARE SYSTEM 10

DESCRIPTION OF PRODUCT :

- a. Inhaling chamber
- b. Nasal nozzle
- c. Receptacle
- d. Power button
- e. Music button
- f. Battery compartment

- If you want to play music to comfort your baby, just push the music start button. Push again to change the music. The music will stop automatically.

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MANUFACTURER :

AVITA CORPORATION No.200.205 Provincial
Free way Song Ling Town,
Wu Jiang City,
Jiang Su, P.R.C. ZIP: 215200

HOW TO INSERT THE BATTERIES :

- Slide the lid of the battery compartment downwards.
- Insert the batteries positioning the positive and negative terminals correctly.
- Close the compartment by sliding the lid upwards.

HOW TO USE THE NASAL CARE SYSTEM 10 :

- Put the nozzle (b) into nostril.
- Turn on to remove nasal secretions and mucus.

NB: Do not tilt head backwards when using the device to ensure it operates efficiently.

- Turn off when finished.

WARRANTY :

This product has a 2-year warranty against defective material and workmanship. During this period, such defects shall be repaired free of charge (proof of purchase should be presented whenever you have a complaint during the warranty period). This warranty does not cover damage caused by accidents, misuse or negligence. In case of complaint, please contact the shop from which you bought the appliance prior to contacting anyone else.

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