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ibp innovative business promotion GmbH, Botzstraße 6, D-07743 Jena, Germany
Hersteller: AVITA Corporation, No.200.205 Provincial Free way, Song Ling Town, Wu Jiang Su, P.R.C.

www.ibp-info.de

Digital blood pressure monitor for readings on the wrist

Instruction manual

1 table of contents

1	table of contents	2
2	important safety information	3
3	the unit / accessories	5
4	display / icons	6
5	batteries / disposal	
	battery installation / replacement	7
	battery safety information	7
	disposal	7
6	setting the date and time	8
7	attaching the wrist cuff	8
8	body posture / blood pressure classification	9
9	instructions for measurement	10
10	the memory function	11
11	an error is displayed / error codes	
	error display	12
	error codes	13
12	important notes	14
13	specifications	15
	service and warranty	16

important safety information 2

- This Blood Pressure Monitor complies with the EC Directives for noninvasive Blood Pressure Monitors. This Blood Pressure Monitor complies with the European regulations and bears the CE mark "CE 0197".
- This Blood Pressure Monitor conforms with the EU Council Directive (EN 1060-1 / EN 1060-3) for noninvasive Blood Pressure Monitors. It also conforms with the EU Council Directive "93/42/EEC of 14th June 1993 on Medical Products", and the EU Council Directive "89/336/EEC for electromagnetic compatibility.
- **ibp** hereby confirms that the product delivered together with this Instruction Manual complies with this Directive. The appropriate conformity declaration is available with **ibp**.
- To ensure accurate measurements, the Blood Pressure Monitor must receive a technical check up every 2 years. If you have any questions about these checks, please contact our Customer Services Department (address on the warranty certificate).

device classification: type BF 

CE 0197

Please read this Instruction Manual carefully to know your blood pressure monitor. Then the monitor will give you many years of smooth service and supply measuring data of high accuracy. Only your physician can interpret your blood pressure values. If you suffer from cardiac arrhythmia, always discuss the blood pressure values with your doctor.

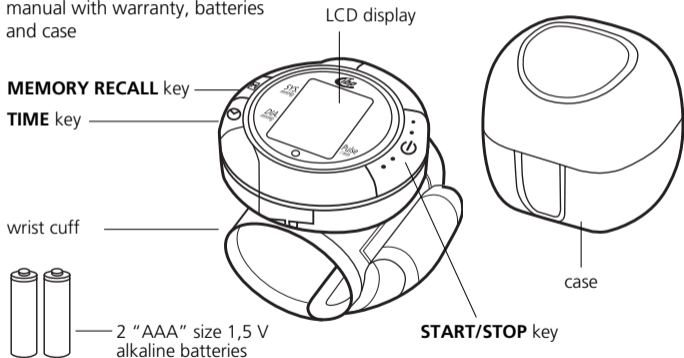
2 important safety information

- Never venture self-diagnosis or self-treatment. The blood pressure should be taken by a doctor or an expert who knows your illness history. By making a regular use of the blood pressure measuring unit and plotting the readings, you keep the doctor up-to-date with the tendencies of your blood pressure.
- Tie the sleeve well around your wrist. The sleeve should be at the same height as your heart. Do not wind the sleeve around the sleeve of your jacket or shirt, otherwise measuring the blood pressure is not possible.
- Do not move or shake the measuring unit during measurement, because that would render the readings inaccurate.
- Carry out the measurement while you are at rest and unstressed.
- Please remember that the blood pressure changes according to nature and the time of the day and also depends upon several factors such as smoking, alcohol consumption, medication consumption and physical stress and strain.
- In case of persons with a state caused by circulation problems (diabetes, kidney problems, calcification of arteries or circulation problems) the readings on this monitor can appear lower than in a blood pressure monitor, which measures the blood pressure on the upper arm. Please consult your doctor, to ensure whether the blood pressure measurement on your wrist will give correct readings.
- The readings obtained with this measuring unit correspond to those of a trained person, who carried out measurement with a sleeve or by listening through a stethoscope.

the unit / accessories 3


identification of the unit

accessories included are instruction manual with warranty, batteries and case


4 the display / icons

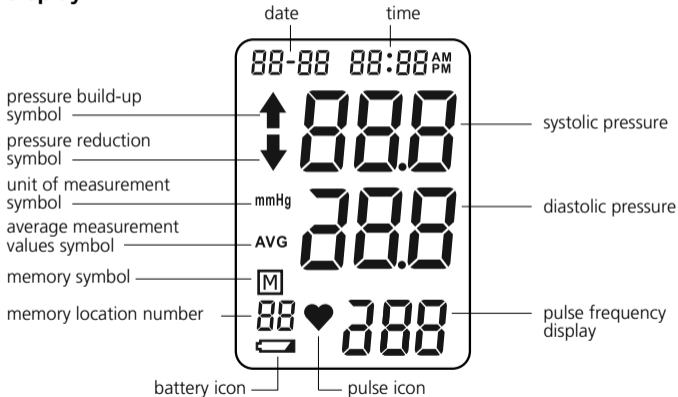display

battery installation / replacement

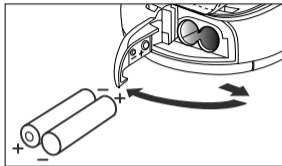Remove the lid from the battery compartment on the underside of the measuring unit and insert the batteries into the battery compartment as shown. Always observe the correct polarity + and - of the batteries. Please only use alkaline batteries!

do not throw batteries away with your household rubbish!

- Please dispose of batteries in accordance with the law! - Batteries are hazardous waste!
- Never short-circuit a battery! - Do not throw the battery into the fire!
- Keep batteries away from children! - Do not attempt to recharge the batteries!

instruction on environment protection

Do not dispose of this product in the usual household garbage at the end of its life cycle; hand it over at a collection point for the recycling of electrical and electronic appliances. The symbol on the product, the instructions for use or the packing will inform about the methods for disposal. The materials are recyclable as mentioned in its marking. By recycling, material recycling or other forms of re-utilization of old appliances, you are making an important contribution to protect our environment.


Please inquire at the community administration for the authorized disposal location.

6 setting the date and time

1. Press the **TIME** key  to enter date and time setting mode. Initially the input display area will flash for the month.
2. To enter the figures for the month, press the **MEMORY RECALL** key **M** until the number of the month you require appears. To confirm, press the **TIME** key  again.
3. Repeat stages 1 and 2 to enter day, hour and minute in succession.

7 attaching the wrist cuff

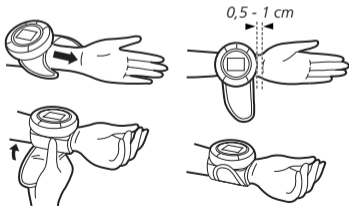1. Wind the sleeve around the wrist. The display of the measuring unit must be on the palm side of the wrist. The wrist must be free.
2. Tighten the sleeve around the wrist. Do not pull forcefully on the sleeve. Do not tighten the sleeve too much.


body posture / blood pressure classif. 8

To obtain as accurate readings as possible, always remember to hold the sleeve at the same height as the heart. If the sleeve is higher or lower than the heart this causes deviations from the actual blood pressure.


measuring blood pressure while sitting

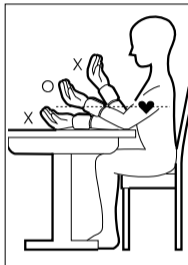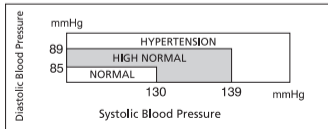1. Support the elbow on a table surface or other surfaces.
2. Place the arm on the arm rest for correct positioning such that the sleeve is at the same height as the heart as far as possible.
3. Relax your hand, the palm must face upwards.

blood pressure classification

The standards for determining blood pressure, regardless of age, are defined by the World Health Organisation (WHO) in the adjacent table.

9 instructions for measurement

- a) Start the Monitor by pressing the **START/STOP** key  .
- b) When the **START/STOP** key  is pressed, all digits will light up. This is a check of the display functions which is complete after about 1 second. The last reading taken will appear in the display.
- c) The unit is ready for use. The unit will automatically start to inflate slowly to measure your blood pressure. During inflation, the pressure build-up symbol  will appear in the display. The unit inflates the cuff until sufficient pressure has built up for a measurement. When pressure reaches to the proximity of your systolic blood pressure, the monitor will exhaust cuff pressure. The pressure build-up symbol in the display will disappear; in its place the pressure reduction symbol  will now appear.
- d) While measuring, do not move the cuff or your arm or hand muscles, don't speak. The cuff will deflate automatically. While the unit determines your pulse frequency, the pulse symbol  will flash.
- e) Once the reading is complete, both the systolic and the diastolic blood pressure, along with the pulse value, will automatically appear in the display. The measurement values are automatically stored in the memory. Up to 60 memories can be saved in the Memory Zone. Pressing the **START/STOP** key  turns off the power. The unit will automatically turn off after about 2 minutes of inactivity.

the memory function 10

- a) You can store your blood pressure reading up to 60 times (blood pressure and pulse) along with the date and time in the memory. The measurement values are stored automatically. Should 60 measurement values already be stored in the memory and a new value is added, the oldest value is deleted.
- b) To retrieve the stored values, press the **MEMORY RECALL** key **M**. The average value (**AVG**) is indicated in the display. If you press the **MEMORY RECALL** key **M** again, the last measurement data stored will be displayed. Each further press of the **MEMORY RECALL** key **M** indicates in numerical order, from 1 to 60, the measurement values previously obtained each time.
- c) If you press the **TIME** key , the values stored will be displayed in reverse order. You have the facility to delete all the data stored. Switch off the unit and hold down the **MEMORY RECALL** key **M** until the display shows "Clr.".
- note:** The data in the memory will not be lost even when the Blood Pressure Monitor is switched off. Replacing the batteries will not affect the memory.

11 an error is displayed

error	fault/cause	solution/explanation
After switching on the unit, the display fails to show anything.	There are no batteries inserted or they have been inserted incorrectly or they are totally empty.	Insert batteries, check the position of the batteries or replace them with new ones.
The change-battery symbol appears. 	The batteries are almost empty or they are completely empty.	Replace both batteries with new ones.
	Batteries may react to frost with a weaker flow of current.	Place the batteries for a short time in a warm place and start again.
The operating time is growing shorter.	You are perhaps failing to use alkaline batteries.	Use alkaline batteries.
No measurement data is displayed.	The voltage of the batteries is extremely low.	Replace batteries with new ones.
Dubious readings are displayed that deviate strongly from the average values.	You have not adopted the correct position.	Adopt the correct position (see Ch. 8, p. 9)
	Blood pressure will deviate from time to time.	
The pulse frequency is higher or lower than the average value.	You have moved your arm or body briefly during the reading or you have spoken.	Do not move while the reading is being taken. Do not talk.
	You have taken the reading immediately after physical exertion.	Do not take any reading immediately after physical exertion.

an error is displayed 11

error	fault/cause	solution/explanation
The unit switches off automatically.	This is built into the system.	Press the START/STOP key again and re-take the reading.
The unit continues to pump up while the reading is being taken.	This is normal. If the user's blood pressure that is being measured is higher than the initial value of the sleeve pressure, the unit will automatically continue to pump. The pressure build-up will occur in stages of 20 mmHg, until a suitable pressure for the reading has been reached.	
	The wrist cuff is not correctly fitted or is not tight enough.	Fit the wrist cuff correctly (see Ch. 7, p.8)

error codes:

Err 0: No pulse or the pulse measured is too low.

Err 1: The sleeve is not airtight or the pressure build-up is too slight.

Err 2: The systolic value cannot be obtained.

Err 3: The sleeve pressure is too high.

Err 4: The diastolic value cannot be recorded or is outside the measuring range.

Err 5: The pressure reduction occurs too quickly.

Err 6: The pressure reduction occurs too slowly.

Err 7: The systolic value is outside the measuring range.

Err 8: The sleeve pressure is outside the measuring range.

Err 9: The sleeve pressure is insufficient, although highest point of the pressure build-up has already been reached.

12 important notes

1. The unit contains high-precision assemblies. Therefore avoid extreme temperatures, humidity, direct sunlight. Do not give strong shock. Do not try to open the blood pressure monitor. This will destroy the calibrated factory settings.
2. Protect from ingress of dust.
3. Do not bend nor twist the preformed cuff. Clean the blood pressure computer body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol as cleaner.
4. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
5. Do not press the **START/STOP** key when the cuff is not placed around the wrist.
6. The unit is determined for household use and should be used by adult persons only.
7. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact the Service Centre.
8. For users diagnosed with common arrhythmia, diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use.

specifications 13

Modell	:	BP 131
Measuring method	:	oscillometric method
Indicator	:	LCD
Measuring range	:	30 - 280 mmHg (blood pressure)
	:	40 - 199 pulse/min. (pulse rate)
Measuring resolution	:	± 3 mm Hg (blood pressure)
	:	±4 % (pulse rate)
Pressure build-up	:	automatic pressure build-up by pump
Pressure release	:	automatic pressure release by valve system
Power source	:	2 x 1.5V = alkaline batteries size "AAA"
Memory	:	60 measuring values
Working temperature	:	+ 10 °C to + 40 °C, humidity 30 - 85 % RH max.
Storage temperature	:	- 20 °C to + 50 °C, humidity 15 - 95 % RH max.
Hard cuff	:	for wrist circumference between 13,5 and 19,5 cm
Dimension	:	appr. 80 (l) x 84 (w) x 33 (h) mm
Weight	:	appr. 130 grams (with batteries)
Explanation of symbol	:	 follow instructions for use!

The right to technical modifications is reserved.

 0197

SERVICE AND WARRANTY

Service Addresses

England: International Business Products Limited

6 Newfield Court West Houghton Bolton BL5 3 SA England Phone: +44(0) 19 42 – 8196 95

India: sano health care products pvt. Ltd. Corporate Office: 28 Landsdown Terrace

1st Floor Kolkata – 700 026 Phone: +91 33 2465 0383-85

Warranty and Compensation for Damages

Starting with the purchase date (date on proof of purchase or receipt) we will perform repairs free of charge for 24 months, should your de-vice be non-operational due to manufacturing or material defects.

Batteries are excluded from the warranty.

The warranty excludes damage resulting from non-compliance or only partial compliance with these instructions for use. Farther-reaching claims such as compensation for damages are likewise excluded.

Please send your device in question as well as the proof of purchase or receipt and the description of the complaint to our service address. Please also attach the warranty documents if you would like to file a warranty claim.

ibp innovative business promotion GmbH, Botzstraße 6, D-07743 Jena, Germany
Manufactured by: AViTA Corporation, No.200.205 Provincial Free way, Song Ling Town,
Wu Jiang Su, P.R.C.

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Fabricante: AVITA Corporation, No.200.205 Provincial Free way, Song Ling Town,