KOMPRESSOR-INHALIERGERÄT

COMPRESSOR NEBULIZER

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COMPRESSOR-INHALEERTOESTEL

INHALADOR PRESURIZADO

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COMPRESSOR NEBULIZER

Instruction manual.
Read all instructions carefully before use.

TABLE O CONTENTS

12 INTRODUCTION

PRODUCT IDENTIFICATION

13 IMPORTANT SAFEGUARDS

3 OPERATING YOUR COMPRESSOR NEBULIZER

16 CLEANING

17 SPECIFICATIONS

18 WARRANTY


INTRODUCTION

Thank you for purchasing the Compressor Nebulizer. It is a compact medical device designed to efficiently deliver physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment.

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

our Compressor Nebulizer should be used under the supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use should always be avoided.

PRODUCT IDENTIFICATION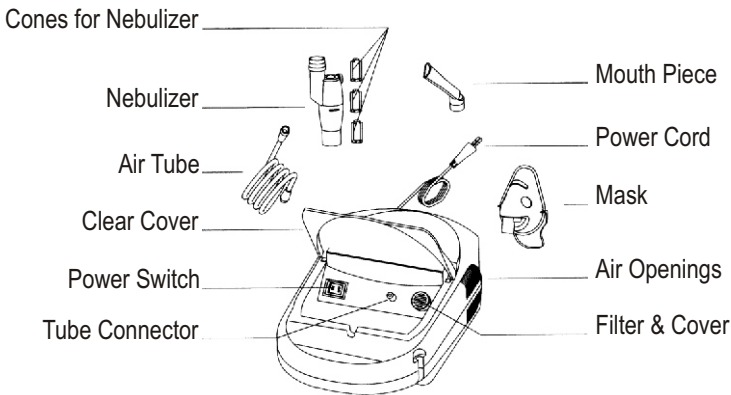

IMPORTANT SAFEGUARDS

Note:

Read all instructions carefully before use.

The following basic precautions are needed when using an electrical product.

Caution:

Failure to read and observe all precautions could result in personal injury or equipment damage.

Product cautions:

1. To avoid electrical shock: Keep the unit away from water. Do not immerse the power cord or the unit in liquid. Do not use while bathing. Do not reach for a unit that has fallen into water - immediately unplug the unit.
2. Never operate the unit if it has any damaged parts (including power cord), if it has been dropped or submerged in water. Promptly send it to a service center for examination and repair.
3. The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
4. Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
5. Do not use other attachments unless recommended by distributor.

Operating cautions:

1. Connect this product to an appropriate voltage outlet for your model.
2. Do not run this product unattended.
3. Never operate if this unit has a damaged cord or plug, if it has been dropped, or in any way, into water, if it does not work properly. Return it to a service center for repair.
4. If any abnormality occurs, discontinue use immediately until the unit has been examined and repaired.
5. Always unplug the product immediately after use.
6. Never block the air openings of the main unit or place it where the air openings may be obstructed.

Storage cautions:

1. Do not store the unit under direct sunlight, high temperature or humidity.
2. Keep the unit out of reach of small children.
3. Always keep the unit unplugged while not in use.

Cleaning cautions:

1. Do not immerse the unit in water. It may damage the unit.
2. Disconnect the unit from the electrical outlet before cleaning.
3. Clean all necessary parts after each use as instructed in this guidebook.

OPERATING OUR COMPRESSOR NEBULIZER

ote:


Prior to initial operation, the nebulizer should be thoroughly cleaned.

1. Place your Compressor Nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated.
2. Open the clear cover and remove the inside accessories.

important:


Prior to initial operation, the nebulizer should be thoroughly cleaned referring to the Cleaning Procedures in the manual.

3. Gently twist the top part of the nebulizer counter-clockwise to disassemble the nebulizer.
4. Consult with your physician to select an appropriate cone for the best treatment. Insert it to the bottom of nebulizer.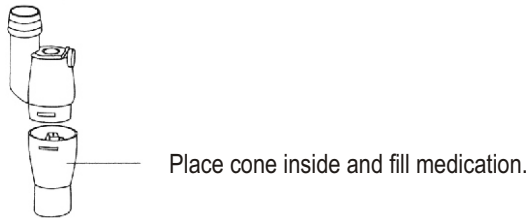

- Cone A for the therapy of the upper respiratory tract
- Cone B for the trachea-bronchial therapy
- Cone C for the therapy of the deep respiration tract

5. Fill the bottom section of the nebulizer with the medication prescribed by your physician.
6. Gently twist the top part clockwise to re-assemble the nebulizer. Be sure the two sections fit well.
7. Attach one end of air tube to the base of the nebulizer
8. Attach the other end of air tube to the tube connector located on the front of the compressor.
9. Attach mouthpiece or mask per your choice to the top section of nebulizer.
10. Plug the power cord into an appropriate electrical outlet. Make sure at this stage, the power switch is at O status.
11. Press power switch to begin your prescribed treatment. Adjust the aerosol outflow by regulating the valve.

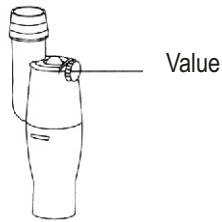Important:

The compressor motor has a thermal protector, which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- a. Switch off the unit.
 - b. Unplug the unit from the electrical outlet.
 - c. Wait 30 minutes for the motor to cool down before another treatment. Make sure the air openings are not obstructed.
12. When treatment is finished, shut off the unit and unplug it from the electrical outlet.

CLEANING

It is recommended that the nebulizer, mouthpiece and mask are thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

Rinsing (after each treatment):

1. Disconnect the air tube, nebulizer, mouthpiece and mask.
2. Gently twist the nebulizer to open it.
3. Rinse the nebulizer, mouthpiece and mask with water.
4. Dry them with clean soft towel or let it air dry.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

Disinfections:

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

1. Use one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece, and mask.
2. Complete Rinsing Steps 1-3.
3. Wash nebulizer, mouthpiece and mask in warm water and a mild detergent. Then wash them in hot tap water.
4. Submerge these parts in the vinegar and water solution for thirty minutes.
5. Complete Rinsing Steps 3-5.

cleaning the compressor:

1. Wipe daily with a damp cloth.
2. Do not use any powdered cleaners or soap pads which may damage the finish.

Filter change:

1. Do not use cotton or any other materials. Do not wash or clean the filter. Only use the filters supplied by your distributor. And do not operate without a filter.
2. Change the filter every 30 days or when the filter turns gray.
3. Changing procedure:
 - a. Remove the filter cover
 - b. Replace the used filter with a new one.
 - c. Replace the filter cover.

SPECIFICATIONS

Power:	AC 230V, 50Hz
Power Consumption:	80 VA
Medication Capacity:	5ml
Particle Size:	0.5 to 10 μm
MMAD:	4 μm
Sound Level:	Around 60 dBA
Average Nebulization Rate:	Min 0.2ml/min
Compressor Pressure Range:	30 to 36 Psi 210 to 250 KPa / 2.1 to 2.5 bar
Operating Pressure:	100KPa / 0.5 to 1.0 bar
Liter Flow Range:	6-8 lpm
Operating Temperature Range:	10 C to 40°C (50 F to 104 F)
Operating Humidity Range:	10 to 95% RH
Storage Temperature Range:	-20°C to 70 C (-4F to 158 F)
Storage Humidity Range:	10 to 95 % RH
Dimension L x W x H:	300 x 180 x 115mm 11.8 x 7.1 x 4.5
Weight:	4.5Lbs 2.1 k
Standard Accessories:	Nebulizer with 3 cones, Air Tube, Mouth piece, 5 Filter, Mask

Protection against electric shock.

Class II Equipment.

Type BF applied parts: Mouth piece & mask.

Degree of safety in the presence of flammable anesthetics or oxygen.

No AP/APG (Not suitable for use in the presence of flammable anesthetics or oxygen)

Mode of operation - continuous.

WARRANT

The Compressor Nebulizer CN-01 M was checked for function before shipment.

To obtain service under the terms of this warranty, mail the product to the address below. Make sure to enclose a note describing the difficulty you have with the monitor and the bill of sale or cash receipt.

All repairs due to defects of workmanship or material will be repaired free within 36 months from the date of purchase (date of the cash receipt/sales bill).

The warranty does not apply if the monitor is used contrary to instructions, handled, stored or cleaned improperly or in any other unusual circumstances.

Any further claims, e.g., claims for damages, are excluded.

If you want to obtain warranty service, mail the monitor and the warranty documents to the customer service of:

International Business Products Limited
6 Newfield Court
West Houghton
Bolton BL5 3 SA England
Phone 44 (0) 19 42 - 81 96 95

sano health care products pvt. Ltd.  - Group
Corporate Office: 28 Lansdown Terrace, 1st Floor
Kolkata - 700 026
Phone No: 91 33 2465 0383-85

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ibp innovative business promotion GmbH