ibp innovative business
promotion GmbH

In accordance with
international standard

ISO **CE**
9001 **0366**

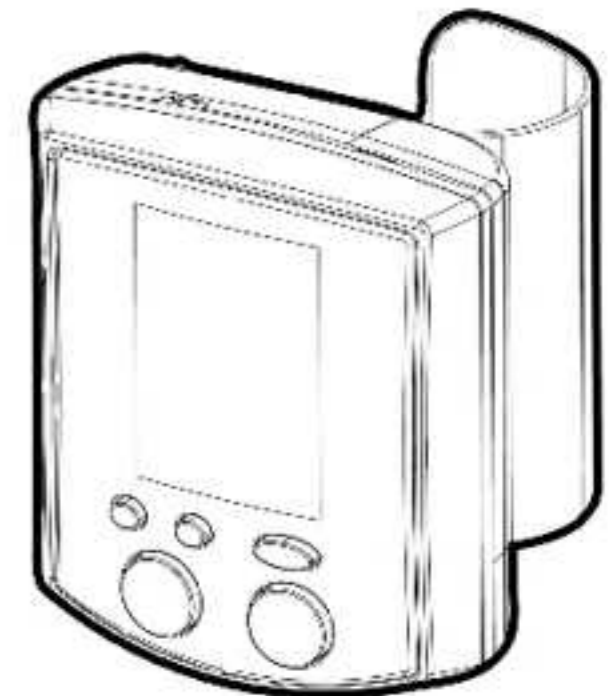
Importer: ibp innovative business promotion GmbH
Address: Botzstrasse 6, D-07743, Jena, Germany
Manufacturer: Rossmax International Ltd
Address: 12F, NO 189, KAN CHIEN ROAD, NEI HU
DISTRICT, TAIPEI, TAIWAN

ibp innovative business
promotion GmbH

Wrist
BLOOD PRESSURE MONITOR
J401

- GB** Instruction Manual
- D** GEBRAUCHSANWEISUNG
- F** MANUEL D'INSTRUCTIONS
- E** MANUAL DE INSTRUCCIONES
- P** Manual de instruções
- PL** Instrukcja Obsługi
- I** MANUALE DI ISTRUZIONI
- NL** GEBRUIKSAANWIJZING
- TR** Kullanım Kılavuzu
- AR** دليل التعليمات
- R** Инструкция по эксплуатации
- C** 使用說明書

GB Introduction

Blood pressure measurements determined with **J401** are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers.

J401 is protected against manufacturing defects by an established International Warranty Program. For warranty information, you can contact the manufacturer, Rossmax International Ltd. or your local distributors.

Attention: Consult the accompanying documents.

Please read this manual carefully before use. For specific information on your own blood pressure, contact your physician. Please be sure to keep this manual.

GB Preliminary Remarks

This Blood Pressure Monitor complies with the European regulations underlying the German Law governing medical products (MPG) and bears the CE mark "CE 0366". The quality of the device has been verified and conforms to the provisions of the EC council directive 93/42/EEC dated 14 June 1993 concerning medical devices, as well as the EMC directive 89/336/EEC:EN 1060-1 (Dec. 1995) Non-invasive blood pressure measuring equipment General requirements and EN 1060-3 (Feb. 1997) Noninvasive blood pressure measuring equipment Supplementary requirements for electromechanical blood pressure measuring systems. To ensure accurate measurements, it is recommended that calibration be completed every two years.

GB How "J401" Works

J401 uses oscillometric method to detect your blood pressure. Before the cuff starts inflating, the device will establish a baseline cuff pressure, which is equivalent to the air pressure. The measurement of your blood pressure is based on this baseline pressure. After the cuff inflates to block your blood in the artery, the deflation process starts. During the deflation of the wrist cuff, **J401** is detecting the pressure oscillations generated by the beat-to-beat pulsatile. Any muscle movement during this period of time will cause measurement error. After detecting the amplitude and the slope of the pressure oscillations during the deflation process, **J401** will determine the systolic and diastolic pressures for you, and your pulse rate is detected at the same time.

Systolic (SYS.)	< 120	120 – 139	140 – 159	≥ 160
Diastolic (DIA.)	< 80	80 – 89	90 – 99	≥ 100
GB	Normal	Prehypertension	Stage 1 Hypertension	Stage 2 Hypertension
D	Normal	Prähypertonie	Hypertension Bluthochdruck, Stufe 1	Hypertension Bluthochdruck, Stufe 2
F	Normale	Préhypertension	Hypertension stade 1	Hypertension stade 2
E	Normal	Prehipertensión	Fase 1 Hipertensión	Fase 2 Hipertensión
P	Normal	Pré-hipertensão	Hipertensão de estágio 1	Hipertensão de estágio 2
PL	Ciśnienie normalne	Ciśnienie podwyższone	Nadciśnienie pierwszego stopnia	Nadciśnienie drugiego stopnia
I	Normal	Pre-iptensione	Stadio 1 Iptensione	Stadio 2 Iptensione
NL	Normaal	Prehypertensie	Fase 1 Hypertensie	Fase 2 Hypertensie
TR	Normal	Prehipertansiyon	1. Evre Hipertansiyon	2. Evre Hipertansiyon
AR	المرحلة الطبيعية	ما قبل زيادة ضغط الدم	المرحلة الثانية من زيادة ضغط الدم	المرحلة الأولى من زيادة ضغط الدم
R	Норма	Предгипертоническое состояние	1 стадия гипертонии	2 стадия гипертонии
C	正常	高血壓前期	第一期高血壓	第二期高血壓

GB

For reliable monitoring and reference of blood pressure, keeping long-term records is recommended. Please download the blood pressure log(chart for recording and tracking crucial info : date, time, blood pressure, pulse rate and body condition) at www.rossmaxhealth.com

GB About Blood Pressure

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard, according to which areas of low- and high-risk blood pressure are identified. This standard, however, is a general guideline as individual's blood pressure varies among different people and different age groups...etc.

It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

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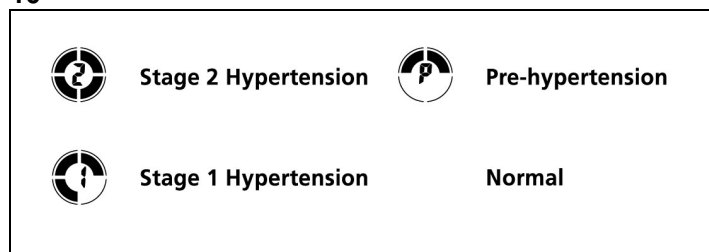
GB Hypertension Classification Indicator

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard according to which areas of low- and high-risk blood pressure are identified, classifying blood pressure ranges into 4 stages. This unit is equipped with innovative blood pressure classification indicator, which visually indicates the assumed risk level (prehypertension / stage 1 hypertension / stage 2 hypertension) of the result after each measurement. (Please note above.)

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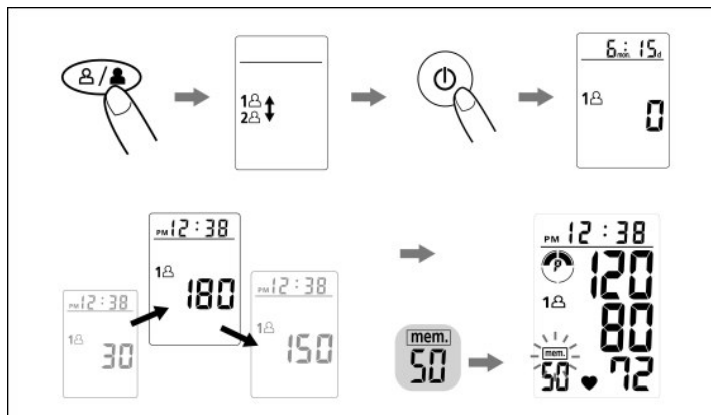
GB Measurement Procedures

1. Press the "User-Switching" key to select a memory zone, and press the "Start" key to start measurement in the chosen memory zone.
2. After checking the display with all digits lighting up, the monitor will automatically inflate.
3. When completed, the systolic pressure, diastolic pressure and pulse rate will be shown simultaneously on the same LCD screen. The values are automatically stored in the pre-designated memory zone.

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GB Cautionary Notes

1. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
2. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
3. The unit should not be operated by children so to avoid hazardous situations.
4. **J401** is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact the store or the doctor from whom you purchased **J401** or please contact Rossmax International Ltd.
5. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device may have difficulty in determining the proper blood pressure.
6. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
7. To stop inflation at any time, press the  key, and the air in the cuff will be rapidly exhausted.
8. Once the inflation reaches 300 mmHg, the unit will start deflating rapidly for safety reason.

Notes:

Batteries are hazardous waste.
Do not dispose them together with the household garbage.

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GB Waste disposal and care of the environment

At the end of its life cycle, this product must not be disposed of in regular domestic waste, but must be taken to a collection site for recycling electric and electronic devices (disposal of used appliances).

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Error Messages

Error Message	Root Causes	Corrections
EE	Measurement Error	Exceeding measurement range. Measure again quietly. Wrap the cuff correctly and keep wrist steady during measurement. If error keeps occurring, return the device to the local distributor or service center.
E1	Air Circuit Abnormality	Measure again quietly. If error keeps occurring, return the device to the local distributor or service center.
E2	Pressure Exceeding 300 mmHg	Switch the unit off and measure again quietly. If error keeps occurring, return the device to the local distributor or service center.
E3	Data Error	Remove and reload the batteries. If error keeps occurring, return the device to the local distributor or service center.

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GB Specifications

Power source	: 3V DC Two LR03 (AAA) Alkaline batteries
Measurement method	: Oscillometric
Measurement range	: Pressure: 40~250 mmHg Pulse: 40~199 beats/minute
Accuracy	: Pressure: ± 3 mmHg Pulse: within $\pm 5\%$ of reading
Pressure sensor	: Semi-conductor
Inflation	: Pump driven
Deflation	: Automatic Pressure Release Valve
Memory capacity	: 50 memories for each zone x 2 Zones
Auto-shut-off	: 1 minute after last key operation
Operation environment	: 10°C~40°C(50°F~104°F); 40%~85% RH max.
Storage environment	: -10°C~60°C(14°F~140°F); 10%~90% RH max.
Weight	: 120g (w/o batteries)
Wrist circumference	: 13.5~22cm (5.3"~8.7")
Dimensions	: 82(L) x 69(W) x 66(H) mm
Limited user	: Adult users
	: Type BF Device and cuff are designed to provide special protection against electrical shocks.

*Specifications are subject to change without notice.

Service and Warranty

Service Addresses:

England: International Business Products Limited, 6 Newfield Court, West Houghton, Bolton, BL5 3 SA England, Phone: +44(0) 1942 – 8196 95
www.ibphealthcare.com

India: sano health care products pvt. Ltd.
 Corporate Office: 28 Landsdown Terrace, 1st Floor, Kolkata 700 026
 Phone: +91 33 2465 0383-85

Warranty and Compensation for Damages

Starting with the purchase date (date on proof of purchase or receipt) we will perform repairs free of charge for 24 months, should your device be non-operational due to manufacturing or material defects.

Batteries are excluded from the warranty.

The warranty excludes damage resulting from non-compliance or only partial compliance with these instructions for use.

Farther-reaching claims such as compensation for damages are likewise excluded.

Please send your device in question as well as the proof of purchase or receipt and the description of the complaint to our service address. Please also attach the warranty documents if you would like to file a warranty claim.

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